

1.0	Description:		delivery, working and setting times of OsteoVation <i>Inject</i> for the particular desired use.
1.1	OsteoVation® <i>Inject</i> Bone Void Filler is an injectable and biocompatible calcium phosphate bone void filler. The single-use OsteoVation <i>Inject</i> Kit contains the necessary components for mixing of the bone void filler.	5.4	The effects of OsteoVation <i>Inject</i> on patients with the following indications are not known:
1.2	The OsteoVation <i>Inject</i> sterile kit contains: Calcium Phosphate Powder, Sodium Silicate Liquid, and a Mixing System (Mixing Bowl, Pestle and spatula).		<ul style="list-style-type: none"> • Documented Renal Disease • Metabolic bone disease • Pregnancy/nursing • Sinus obliteration • Infection during the last three months • Radiation and/or chemotherapy post-implantation • Defects due to disease or congenital malformation • Cardiovascular disease precluding elective surgery • Patients less than 18 years of age
2.0	Indications:		
2.1	OsteoVation <i>Inject</i> is a calcium phosphate bone void filler indicated for the repair or filling of neurosurgical bur holes, other craniofacial defects and craniotomy cuts with a surface area no larger than 25cm ² . OsteoVation <i>Inject</i> may be used in the restoration or augmentation of bony contours of the craniofacial skeleton, including the fronto-orbital, malar, and mental areas; it is non-suitable for use in kyphoplasty procedures. The product provides a bone void filler that resorbs and is replaced by bone during the healing process.		
3.0	Contraindications:	6.0	Adverse Reactions:
	OsteoVation <i>Inject</i> is not designed or sold for any use except as indicated. DO NOT use OsteoVation <i>Inject</i> IN THE PRESENCE OF ANY CONTRAINDICATION:		A successful result is not achieved in every single surgical case. Reoperation to remove or replace the implant may be required at any time due to medical reasons or device failure. If corrective action is not taken, complications may occur. The occurrence of any of the following applications is possible and may require reoperation and/or removal of the implant.
3.1	Use in any infected site or surgical site located near an infection.		Complications may include but are not limited to:
3.2	Use for stress-bearing applications (e.g. mandibular segment replacement)	6.1	Tissue thinning over implant site.
3.3	Use in areas where surrounding bone is not viable, or is incapable of supporting or anchoring the implant.	6.2	Tenderness/redness/edema.
3.4	Use in patients who have not reached an age in which skull growth is essentially complete.	6.3	Seroma/hematoma or infection
3.5	Use in patients with an abnormal calcium metabolism, metabolic bone disease, a recent untreated infection or immunological abnormalities.	6.4	Swelling/fluid collection/Infection of the soft tissue and/or bone (osteomyelitis) and fever
3.6	Use in patients, who undergo or are to undergo an immunosuppression therapy.	7.0	Sterility:
3.7	Use in patients with acute traumatic injuries with open wounds close to the defect which are likely to become infected.	7.1	All components of the OsteoVation <i>Inject</i> are provided sterile and are intended for single use only. Do not re-sterilize.
3.8	Use in patients having received or are to receive chemotherapy or radiation therapy at or near the implant site.	7.2	Sterile product packaging should be inspected for flaws and integrity prior to opening. In the presence of such a flaw, the product must be considered non-sterile and appropriately discarded.
4.0	Warnings:	8.0	Storage:
4.1	OsteoVation <i>Inject</i> attains a physiological pH after components are properly mixed - the Calcium Phosphate Powder and Sodium Silicate Liquid components may be irritants separately.	8.1	The OsteoVation <i>Inject</i> Kit is to be stored at a controlled room temperature between 15°C to 30°C (59°F to 86°F).
4.1.1	Skin Exposure: Wash exposed area with soap and water. Seek medical attention if irritation develops.	9.0	Instructions for Use:
4.1.2	Eye Exposure: Flush thoroughly with running water. Seek medical attention if irritation develops.		The operating physician should be experienced in current advances in surgical techniques and standard operating procedures. Additional training from a company representative is recommended.
4.2	OsteoVation <i>Inject</i> is a single use only product; unused portions of OsteoVation <i>Inject</i> Bone Void Filler must be discarded and cannot be re-sterilized. NOTE: All components of OsteoVation <i>Inject</i> must be cooled to at or below 21°C (70°F) prior to use. NOTE: Attempts to reuse or re-sterilize OsteoVation <i>Inject</i> may result in patient harm and may expose the patient to the risk of transmitting infectious disease. NOTE: Before disposal of any unused OsteoVation <i>Inject</i>, mix the Calcium Phosphate Powder and the Dilute Sodium Silicate Liquid together to render the material harmless to personnel and the environment.	9.1	Implantation of OsteoVation <i>Inject</i> should be performed under sterile or aseptic operating room conditions and image intensification is advised.
5.0	Precautions:	9.2	Implant Site Preparation: Remove blood clots and tissue debris, lavage and suction may be used.
5.1	Use only the components provided with the OsteoVation <i>Inject</i> Kit for the mixing of OsteoVation. Substitution of components may alter the desired results.	9.3	Proper eye protection and surgical gloves must be worn when mixing OsteoVation <i>Inject</i> .
5.2	The paste remains injectable and implantable for up to 5 minutes at room temperature (19°C to 21°C, 66°F to 70°F). Once the paste has been introduced into the body, it will remain workable for 2 minutes at 32°C (90°F). Following this 2 minute working period, the material and the site should not be disturbed for an additional 4 minutes to allow for proper initial setting. The material will continue to cure in the next 24 to 72 hours after implantation to reach its maximum strength.	9.4	Mix OsteoVation <i>Inject</i> by pouring the Calcium Phosphate Powder into the mixing bowl, while tapping the vial to ensure maximum transfer of powder. Pour the liquid into the mixing bowl, while tapping the vial to ensure maximum transfer of liquid.
5.3	The medical professional is responsible for using his/her best medical judgment when using this device. Prior to the implantation of this bone void filler, the medical professional should develop a preoperative plan which takes into account the mixing,	9.5	Using the pestle, mix the Powder and Liquid together for approximately 1 minute using a circular stirring motion. NOTE: During the mixing process material may collect on the pestle. This material must be reincorporated into the mixing process to ensure a proper mix of OsteoVation <i>Inject</i>.
		9.6	Once mixed, OsteoVation <i>Inject</i> may be delivered to the targeted site using manual impaction or injection. If delivery via injection is desired, use the sterile spatula to scoop and load OsteoVation <i>Inject</i> out of the mixing bowl and into either a sterile delivery syringe supplied by OsteoMed or a commercially available sterile delivery syringe. NOTE: The efficacy of alternative delivery methods other than those recommended or supplied by OsteoMed has not been evaluated.

- 9.7 OsteoVation *Inject* will remain injectable for approximately 5 minutes at standard operating room temperatures (19°C to 21°C, 66°F to 70°F).
- 9.8 If injecting OsteoVation *Inject*, utilize a retrograde approach to fill the void
- NOTE: Operating room temperatures above 21°C (70°F) may adversely affect product performance, specifically mixing and injection. If the operating room temperature is above 21°C (70°F) the entire OsteoVation kit should be cooled to below operating room temperature, but not to freezing temperatures.**
- 9.9 OsteoVation *Inject* will remain workable for 2 minutes following implantation.
- 9.10 After the above described working time, OsteoVation *Inject* and the operative site should not be disturbed for an additional 4 minutes to ensure proper initial setting.
- NOTE: Initial set up time for OsteoVation *Inject* is 6 minutes following implantation at 32°C (90°F).**
- NOTE: If screw insertion is desired, a pilot hole should be placed in OsteoVation *Inject* following the post implantation setting period of 6 to 12 minutes (6 minutes if OsteoVation is not exposed to air; 12 minutes if OsteoVation is exposed to air) with subsequent screw insertion (1.6 or 2.0mm). Final tightening of the screw can occur after an additional 5 minutes. During final tightening the screw head should remain flush with the surface of OsteoVation, not any deeper.**
- 9.11 Following the 6 minute setting period, implants may be placed and then closure of the surgical site may be commenced. The ultimate strength of the material is reached 24 to 72 hours post implantation
- 9.12 OsteoVation *Inject* is designed to set in a wet surgical environment at body temperature. Initial and hard setting periods will be delayed if the temperature of the surgical implantation site is equal or less than 32°C (90°F).

Distributed by
OSTEOMED L. P
3885 Arapaho Road
Addison, Texas, 75001
972-677-4600
www.osteomedcorp.com

The contents of this package are covered under patent 6,375,935, 6,719,933 and other pending patents

Skeletal Kinetics® and its logo are registered trademarks of Skeletal Kinetics, LLC.

STERILE R



Federal (USA) Law
restricts this device to
sale by or on the
order of a physician.

CE
0050

EC REP



Skeletal Kinetics LLC
10201 Bubba Road
Cupertino, CA 95014-4167
USA
408-366-5000

BB1 – Trade B.V
Koekoeklaan 20
1343 AM Almere
The Netherlands

030-1404, Rev. E
LBL 10522 - AI



Inject Bone Void Filler Instructions for Use