

PROHESIONTM

Hydrolyzed Surgical Powder

Indications for Use:

For use in the management of surgical wounds, traumatic wounds, first and second degree burns and superficial wounds.

Device Description:

Prohesion is approximately 95% Type 1 collagen. It does not contain any preservatives, alcohol or synthetic substitutes. The patented* collagen fragments are a fraction of the size of collagen molecules found in native, intact collagen.

When to Use:

Moderately to heavily exudative wounds. Powder will absorb up to 30 times its weight in fluid and contains the highest percentage of collagen.

Contraindications:

Known sensitivities to bovine or collagen products.

Initial Application:

It is recommended that the wound bed be debrided of slough and necrotic tissue. Cleanse the wound site per protocol. Apply Prohesion to the wound bed. The product should cover the entire wound bed and the edges of the wound. Cover with the appropriate dressing that maintains optimum moisture level of the wound bed. Note: Infected wounds should be treated as per facility protocol. Prohesion may be used with other dressings.

Reapplication of Prohesion:

Prohesion does not need to be removed from the wound site. As applicable, re-apply Prohesion (2-3) times per week or with each dressing change.

Storage Conditions:

Store at room temperature and protect from freezing.

Shelf Life and Disposal:

The expiration date is printed on the label. DO NOT USE Prohesion AFTER THE EXPIRATION DATE.

*Protected by US Patent 6,136,341

DO NOT USE IF PACKAGE IS DAMAGED.

STERILE

Sterile Until Open



Single Use Only



See Instructions
For Use

PH110

Hydrolyzed Surgical Powder

Distributed by:

 **BIOSTRUCTURES[®]**

Newport Beach, CA 92660 . Ph: 949.553.1717 . Fx: 949.553.0407

©2014 BioStructures, LLC
255870