



EC REP
Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands



 Marine Polymer Technologies, Inc.
107 Water Street
Danvers, MA 01923 U.S.A.
1-888-666-2560

US Patent Numbers
8,486,033 6,743,783 5,858,350
8,481,512 6,686,342 5,846,952
8,152,750 6,649,599 5,686,115
7,931,637 6,630,459 5,635,493
7,285,266 6,610,668 5,624,679
7,115,588 6,599,720 5,622,834
7,041,657 6,063,911 5,623,064
6,864,245

MADE IN USA
MPS 500-70 Rev 04 09/2014

mr^odh[®]

A trauma dressing intended for the temporary control of severely bleeding wounds such as surgical wounds (operative, postoperative, donor sites, dermatological, etc.) and traumatic injuries.

REF 400-25 (5cm x 5cm)

REF 400-14 (10cm x 10cm)

DESCRIPTION: mrdh® is a soft, white, sterile, non-woven, lyophilized pad of a cellulosic polymer isolated from microalgae (poly-N-acetyl glucosamine). It is attached to x-ray detectable gauze and packaged in a sterile blister pack.

INDICATIONS: mrdh® is intended for use under the direction of a healthcare professional.
mrdh® is a trauma dressing intended for the temporary control of severely bleeding wounds such as surgical wounds (operative, postoperative, donor sites, dermatological, etc.) and traumatic injuries.

PRECAUTIONS:

There are no known contraindications to the use of mrdh®.

mrdh® APPLICATION:

1. Open sterile blister pack.
2. Place mrdh® on open wound, with active side toward wound surface.
3. Apply firm hand compression until bleeding stops.
4. If the dressing becomes completely soaked through, apply another dressing over the first and continue pressure.

TO REMOVE:

1. Soak mrdh® with sterile saline.
2. Gently peel away.

CAUTION:

Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

Not Made with Natural Rubber Latex

Sterilized with radiation.

Do not resterilize and/or reuse.

Single Use Only.