

# MAINTENANCE AND WARRANTY GUIDE

## AQUAROLL® A2S2 SYSTEM



### INITIAL INSTALLATION

Professional installation of the AquaRoll® A2S2 system is highly recommended. Service and installation contracts are available, upon request.

- **Caution:** Glass beads must be thoroughly rinsed three (3) times with four (4) liters of water for one (1) minute per rinse prior to initial client use of the AquaRoll® A2S2 system.
- **Caution:** Only use glass beads supplied by AquaRoll®, as they are specially designed for full functionality.
- **Caution:** Do not exceed two and a half (2.5) bags of glass beads at one time in the system.
- **Note:** Fewer beads may be used when treating smaller extremities. The maximum quantity of beads may be utilized when treating large feet, such as with male athletes.

### STORAGE REQUIREMENTS

Always press the “On/Off” power switch to “Off” and unplug the device after each daily usage session.

Store the AquaRoll® A2S2 system in a moderate, dry location.

- **Caution:** Never store the AquaRoll® A2S2 system in a location with a temperature below ten (10) degrees Celsius.
- **Caution:** Never allow unattended children or mentally disabled clients around the AquaRoll® A2S2 system, as there is a risk of ingestion of glass beads.

It is recommended to cover the device, when not in use.

Where necessary to discard the AquaRoll® A2S2 system, ensure full compliance with local waste regulations.

### MAINTENANCE

The drainage valve of the tank must be cleaned quarterly, at a minimum. Refer to the AquaRoll® Care and Cleaning Instructions (90000007) for further details.

Glass beads occasionally require replacement after several thousand treatment sessions. Consult an AquaRoll® service professional for further details.

- **Caution:** Failure to replace the glass beads can result in decreased client benefit, due to the loss of uniformity and constancy of the percussive hydro-massage. Such failure can also result in worn-down beads getting stuck inside the drainage and system tubing and can void the warranty.

### WARRANTY INFORMATION

The AquaRoll® A2S2 system comes with a one (1) year warranty on parts.

**The warranty is void upon the following conditions:**

- **improper use and/or evidence thereof**
- **attempted opening and/or repair of the machine by anyone other than the designated service professional**

Additional parts and labor warranties are available for purchase, upon request. Please contact AM Rejuvenation for details.

---

#### AM Rejuvenation, LLC

6538 Collins Avenue, #313  
Miami Beach, FL 33141

To order or for info, call:  
888.850.3797

Find more info online:  
[www.aquarollUSA.com](http://www.aquarollUSA.com)

Request info by email:  
[sales@aquarollUSA.com](mailto:sales@aquarollUSA.com)