3. Injection

Administer medication in accordance with established protocol. For user convenience, the needle "bevel-up" position is oriented to the safety cover.



4. Activation

After injection, immediately activate safety cover as follows:

- Using the same hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- Push the cover forward toward the needle until you hear and/or feel it lock. Visually confirm that the needle tip is covered.



Discard after single use in an approved sharps container.

Don't Forget...

Aseptic technique and proper skin preparation of the site are essential. Follow your institution's policy for safe disposal of all medical waste.

BD Eclipse™ Needle for Luer Lock Syringes - 100/Box – 1200/Case				
Catalog #	Needle Size	Catalog # N	eedle Size	
305761	25G x 1"	305765	21G x 1½"	
305767	25G x 1½"	305766	18G x 1½"	
305762	23G x 1"	305759	25G x ⁵ /8"	
305768	22G x 1"	305758	27G x ½"	
305763	22G x 1½"	305757	30G x ½"	
305764	21G x 1"			



305764	21G x 1"				
BD Eclipse™ Needle with Luer Lock Syringe 1mL BD Luer -Lok™ Syringe with Detachable Needle - 50/Box - 300 Case					
305778	30G x ½"	305780	25G x ⁵ /8"		
305789	27G x ½"				
3mL BD Luer -Lok™ Syringe with Detachable Needle - 50/Box - 300 Case					
305781	25G x 5/8"	305783	22G x 1½"		
305787	25G x 1"	305779	21G x 1"		
305782	23G x 1"	305784	21G x 1½"		
305788	22G x 1"				

5mL BD Luer -Lok™ Syringe with Detachable Needle - 50/Box - 300 Case				
305785	22G x 1 ½"			
10mL BD Luer -Lok ^{**} Syringe with Detachable Needle - 50/Box - 300 Case				
305786	22G x 1½"			

REMEMBER: Safety-Engineered Devices are required in your Custom and Standard Procedure Trays.

For more information call 1.888.237.2762

BD. BD Logo and all other trademarks are property

of Becton, Dickinson and Company. © 2013 BD.

BD 10/13-MSS0078 - Rev1







BD Eclipse™ Needle

How to Use the BD Eclipse™ Needle for Luer Lock Syringes

1. Attachment

- 1a. Firmly attach needle onto a Luer Lock syringe with a push and clockwise twist.
- 1b. Pull back on the safety cover toward the syringe and away from the needle.
- 1c. Grasp the syringe with one hand and with the other hand pull the clear needle shield straight off.



2. Aspiration

Draw up medication as per your usual technique.

NOTE: If changing needles, activate safety cover prior to removing the needle from the syringe.



The BD Eclipse™ Needle offers clinicians a wide range of features and benefits for luer lock applications.



After Activation



Designed With Important Safety Features And Backed By BD's Reputation For Quality

BD PrecisionGlide™ Needle Technology

- Designed for patient comfort
- Leading brand of hypodermic needle technology in the U.S.

Safe, One-Handed Activation

- Single finger stroke activation: no hard surface required
- Textured finger pad keeps fingers behind the needle
- Requires minimal change in injection technique

Designed To Protect Healthcare Workers From Accidental Needlestick Injuries

- Safety cover is present between clinician and needle
- Audible click indicates safety activation
- Needle remains locked inside activated cover
- The BD Eclipse[™] Family offers one safety technology for both your injection and blood collection needs