

ProClense™ plus

Instrument Detergent

Non-Foaming



***Economical and effective
cleaning solution.***

ProClense plus is an economical detergent with effective cleaning agents and anti-corrosive agents to help maintain instrument surface quality.

Protects Valuable Instrumentation:

- Moderate alkalinity meets AAMI recommendations for effective cleaning and material compatibility. Safe for use on stainless and carbon steel and copper.
- Formulated for use in hard water conditions. High levels of chelating and free-rinsing agents ensure best cleaning results and prevent re-deposition of soils.

Versatile:

- Non-foaming formula is effective for use in automatic instrument washers and cart washers.
- Mild clean scent and no added dyes.



ProClense plus Description	Case Qty	Order#
1 Gallon	4	PCNF128
5 Gallon Drum	1	PCNF050
15 Gallon Drum	1	PCNF150
30 Gallon Drum	1	PCNF300
55 Gallon Drum	1	PCNF550

ProClense™ plus Instrument Detergent

Description:

ProClense plus is a non-foaming, multi-purpose, concentrated cleaner for surgical grade stainless steel, titanium coated steel, copper and brass instruments. ProClense plus is formulated primarily as an instrument cleaner, but it also conditions metal-to-metal parts. ProClense plus is intended to clean and condition surgical grade stainless steel instruments in manual instrument cleaning systems, ultrasonic cleaners and automated washers or cart washers.

Product Specifications:

Dilution Ratios:	1 oz. per gallon of water in manual and ultrasonic systems. ½ oz. to 1 oz. per gallon of water in automated washer/disinfectors based on water quality.
pH:	9.8
Active Ingredients:	Surfactants and cleaning agents, metal surface conditioning agents, chelating, anti-redeposition and anti-corrosive agents.
Temperature of Use:	In manual applications use 90° to 120°F (32° to 49°C). In automated systems follow washer manufacturer's instructions or settings. Best results obtained at 120° to 160°F (49° to 71°C).
Solution Color:	Clear.
Odor:	Non-specific.
Shelf Life:	3 year shelf-life, expiration date printed on product.

Product Capabilities:

ProClense plus, in one step, will quickly loosen and remove organic soils from instruments, reduce mineral deposits and reduce corrosion while conditioning metal surfaces to improve the function of jointed and hinged items.

ProClense plus is non-foaming and ideal for use in high pressure, high impingement instrument washers and cart washers.

Use Precautions:

ProClense plus is not intended for use with lensed equipment. Do not soak items in ProClense plus for prolonged periods (over two hours) or overnight. ProClense plus is a mild irritant. Prolonged skin contact may cause mild irritation or redness. Wear safety goggles or face shield and water resistant gloves when using ProClense plus.

Accessory Products:

Other products available from Certol include:



ProEZ foam™
Ready-to-Use Foaming
Enzymatic Spray



ProEZ AW Quad™
Quadruple Enzyme
Automatic Washer
Detergent



ProLube™ Concentrate
Instrument Lubricant
Concentrate



ProLube™ RTU
Instrument Lubricant
Ready-to-Use