

Set up for Snare/Hot Biopsy Use


1. Turn ON the Unit.
2. Scroll through the programs and select *highlighted* Snare / Hot Biopsy Program.




3. Press the FOCUS Active Display button beside the Monopolar receptacle.



4. Check the Program Screen to confirm the settings (Default program shown).

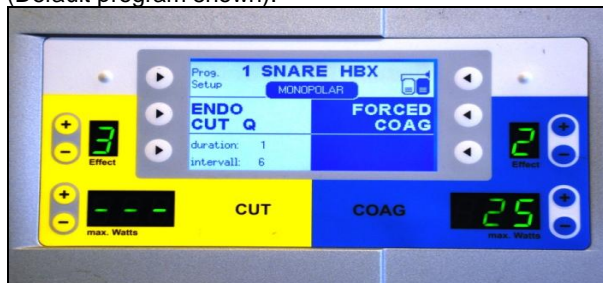

5. Plug in the Active Cord and attach the Snare or Hot Biopsy Forceps.

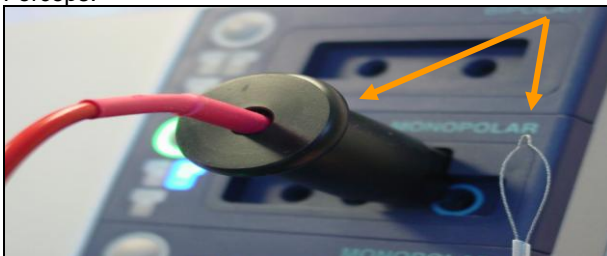


6. Plug the Grounding Pad into the NEUTRAL receptacle and place the Pad on the patient.
7. Place Footswitch at the physician's feet.
8. **Verbally confirm settings with physician.**
8. READY to begin procedure.

Adjust settings per physician direction, increasing the Effect then max.Watts for more power

Set up for Bipolar Use


1. Turn ON the Unit.
2. Scroll through the programs and select *highlighted* Bipolar Probes Program.

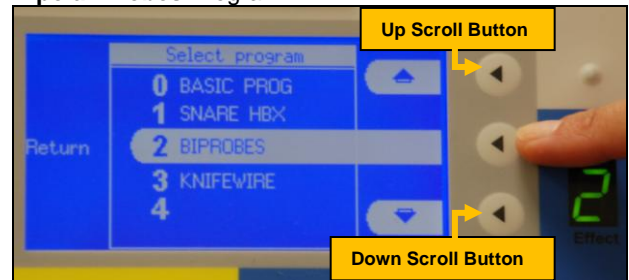

3. Press the FOCUS Active Display button beside the Bipolar receptacle.



4. Check the Program Screen to confirm the settings (Default program shown).

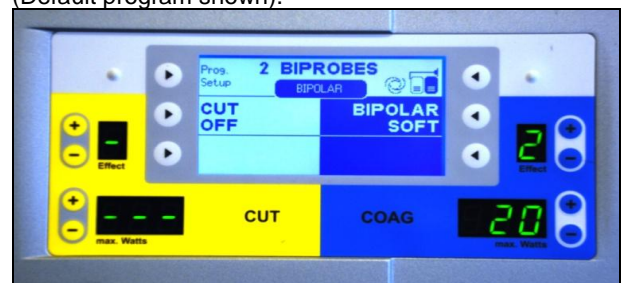

5. Plug in the Adaptor¹ and attach the Bipolar Probe or use a banana style plug probe².



1



2

6. NO Grounding Pad needed.
7. Place Footswitch at the physician's feet.
8. **Verbally confirm settings with physician.**
8. READY to begin procedure.

Adjust settings per physician direction, increasing the Effect then max.Watts for more power.