

Sparkle V™ Technical Data Sheet

PRODUCT DESCRIPTION: Sparkle V is a single-use 5% sodium fluoride cavity varnish with Xylitol that delivers the maximum allowed fluoride dosage. Sparkle V is gluten, aspartame and saccharin free and FDA approved for treating dental sensitivity.

INDICATIONS:

- Effective in the prevention of tooth decay when used in conjunction with a comprehensive **oral hygiene program**.
- Helps prevent tooth sensitivity when applied to exposed roots.
- Prevention of root decay in older patients who experience difficulty cleaning their teeth.

COMPOSITION:

- Rosin
- Denatured Alcohol
- Beeswax
- Sucralose Powder
- Sodium Fluoride
- Dye
- Flavoring
- Xylitol

INSTRUCTIONS FOR USE: These are included in every package of Sparkle V.

STORAGE: Store at or below 73F.

SHELF LIFE: Sparkle V has a 2 year shelf-life.

QUESTIONS & ANSWERS:

- Q** What advantage does Sparkle V 5% Sodium Fluoride Varnish with Xylitol have over other fluoride varnishes?
- A** Sparkle V is nearly invisible after application to the tooth surface. Sparkle V does not clump and is easy to brush onto the tooth and spreads easily. Sparkle V dries in seconds on contact - no worry of fluoride ingestion.
- Q** What is the amount of ppm fluoride in Sparkle V?
- A** 22,600 ppm NaF.
- Q** How often should I apply Sparkle V?
- A** Every six months. Every three months for high risk patients.
- Q** Will my patients notice Sparkle V on their teeth?
- A** Most likely they will detect a film.
- Q** How long should my patients leave Sparkle V on their teeth?
- A** Four to six hours.
- Q** Can my patients eat after application of Sparkle V?
- A** Patients should be advised to remain on a soft food diet and only drink cool liquids for two hours after treatment. Do not brush for a minimum of four to six hours.
- Q** Should patients stop using fluoride rinses or supplements after application of Sparkle V?
- A** The use of fluoride tablets and rinses should be interrupted for several days after initial treatment.
- Q** Are there any contraindications to the use of Sparkle V?
- A** Children with a low risk of cavity formulation. Ulcerative gingivitis. Known allergic reactions to fluoride.



SUMMARY:

- Q** Can Sparkle V be indicated for use as a fluoride-containing coating that treats hypersensitive teeth and exposed dentin and root surface sensitivity?
- A** Yes, this is still a procedure that needs to be repeated every six months.
- Q** Does Sparkle V contain 22,600 ppm fluoride in a unique solvent system?
- A** Yes.
- Q** Is Sparkle V saliva and moisture tolerant as well as saliva and moisture activated?
- A** Yes.
- Q** Does Sparkle V create a physical barrier by covering and occluding dentinal tubules?
- A** Yes.
- Q** Does Sparkle V create a chemical barrier by forming insoluble minerals within dentinal tubules?
- A** Yes.
- Q** Does Sparkle V adhere to teeth to which it has been applied?
- A** Yes.
- Q** Does Sparkle V migrate to tooth surfaces including hard to reach areas?
- A** Yes.

CROSSTEX®
A CANTEL MEDICAL COMPANY

PROTECTSsm

Tel: 631-582-6777 | Toll Free: 888-CROSSTEX | Fax: 631-582-1726
Website: www.crosstex.com | E-mail: Crosstex@crosstex.com

California / Georgia / New York / Pennsylvania / Argentina / Japan / The Netherlands