

# Technical Bulletin


## QuickVue® Influenza Test and QuickVue® RSV Test: Test for Both Influenza and RSV Using One Patient Sample

*One patient sample is sufficient to perform both the QuickVue Influenza and QuickVue RSV tests.*

### Liquid Nasal Specimens

Following your Institution's protocol for obtaining liquid nasal specimens, use a specimen container to collect 650-1000 µL of patient sample. Then follow the sample preparation procedure shown below in Figure 1 and the test procedures in the QuickVue Influenza and QuickVue RSV package inserts.

**Figure 1: Sample Preparation for Wash and Aspirate Specimens**

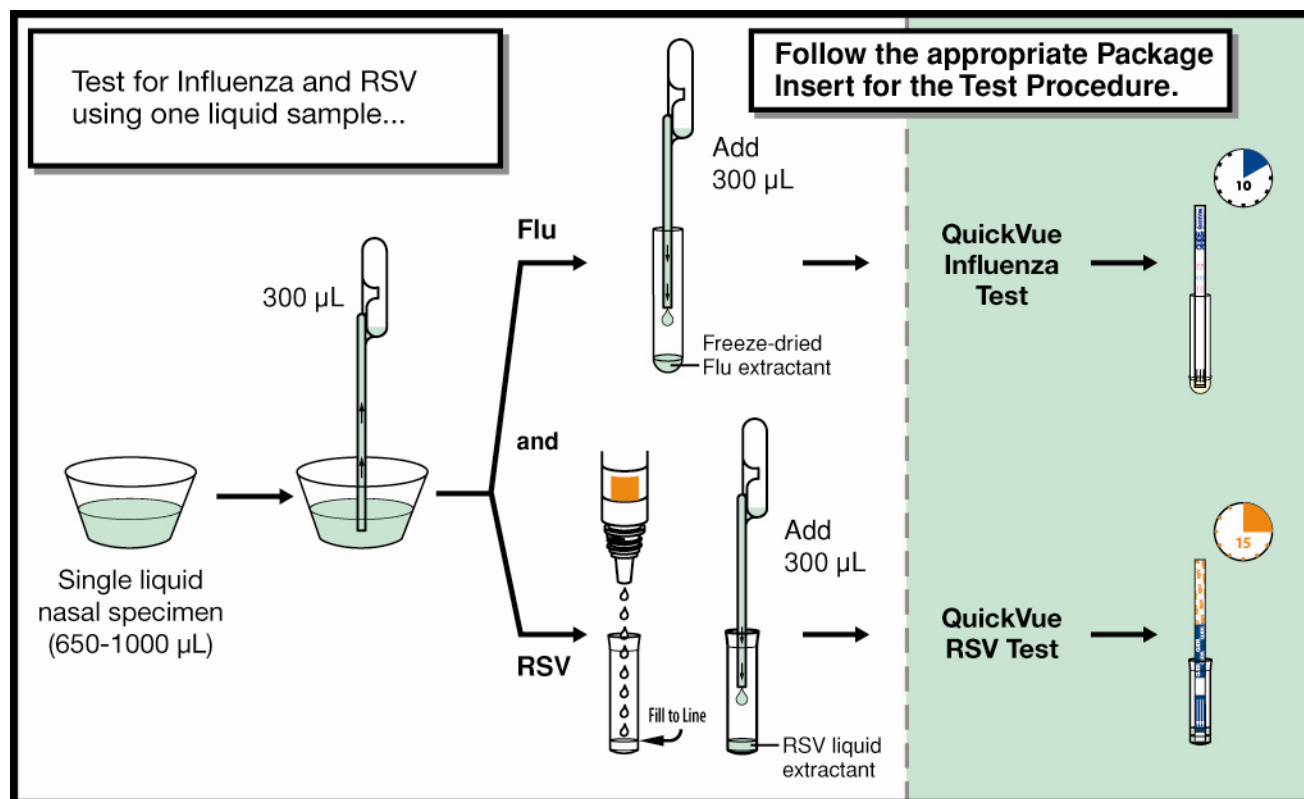

Please contact Quidel Technical Support at 800-874-1517 (USA only), 858-552-1100 or [technicalsupport@quidel.com](mailto:technicalsupport@quidel.com) if you have any questions regarding the QuickVue Influenza A+B test, QuickVue RSV test or any other Quidel product. Our hours of operation are Monday-Friday, 7:00 a.m.-5:00 p.m. Pacific Time.

**If necessary, please contact Quidel Technical Support for specimen tubes or saline vials.**

You may also visit our website at [www.quidel.com](http://www.quidel.com) and [www.flutest.com](http://www.flutest.com) for information on Quidel's line of Rapid Diagnostics, Bone Health and Autoimmune & Complement product lines. Other product information available on our website includes: CPT codes, CLSI, MSDS and package inserts.