

LAPRA-TY® Suture Applier

Optimal Device Performance Guide




LAPRA-TY® Suture Clips are not intended for use in clinical situations that require the use of suture material other than sizes 2-0, 3-0 or 4-0 Coated VICRYL® (polyglactin 910) Suture. Failure to use indicated suture type and size may result in insufficient clip security.


For all product details, including a complete list of warnings and precautions, please reference the Instructions for Use

LAPRA-TY® Suture Applier

Optimal Device Performance Guide

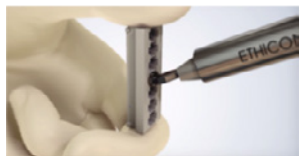

Grasp the applier by the handle and insert the jaws of the instrument into the individual cartridge slot. Ensure the tips of the applier are perpendicular to the surface of the cartridge.

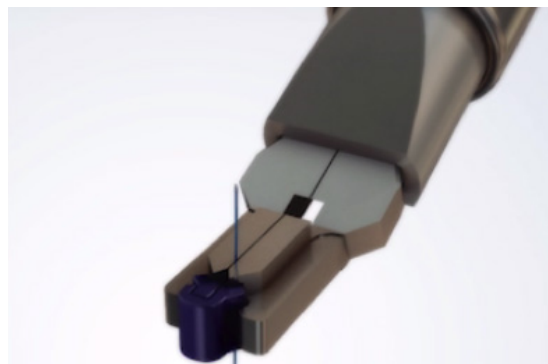


Place a LAPRA-TY® Suture Clip approximately 5mm from the cut end of the suture. **NOTE:** Avoid placing suture near the latch in order to ensure latch security.




LAPRA-TY Suture Clips are not intended for use in clinical situations that require the use of suture material other than sizes 2-0, 3-0 or 4-0 Coated VICRYL® (polyglactin 910) Suture. Failure to use indicated suture type and size may result in insufficient clip security.



After placing the suture using standard continuous or interrupted technique load an additional clip and pass the loaded LAPRA-TY Suture Applier through the trocar.

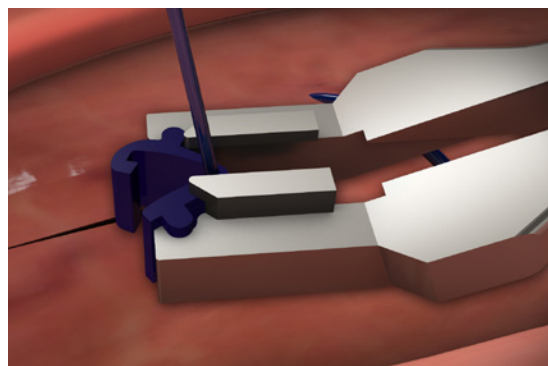

Position the clip by sliding it down the suture until it makes contact with the tissue. **NOTE:** Excessive tension may cause clip slippage.

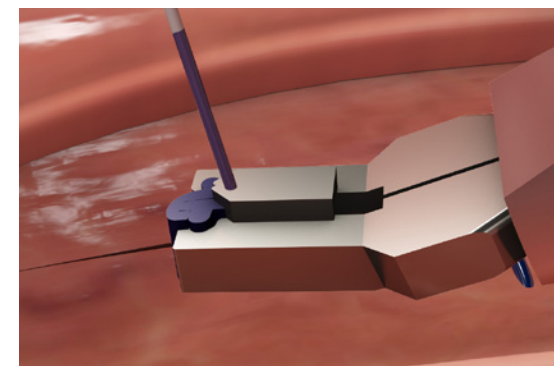

Close the applier jaws by squeezing the handle of the applier until the clip is visibly locked. Surgeon should verify closure and security.

For all product details, including a complete list of warnings and precautions, please reference the Instructions for Use