

MAJOR 

EFFERVESCENT DENTURE CLEANSER

CLEAN WITH THIS PRODUCT
AT LEAST ONCE A DAY TO:

- Kill the bacteria that cause denture odor.
- Dissolve away stains that brushing leaves behind.
- Reduce plaque buildup and remove food particles that can cause gum irritation.
- Keep dentures fresh and clean.



1.

Drop one tablet into a glass or denture bath containing enough very warm (not hot) water to completely cover the denture.




2.

Immediately place denture or bridge in effervescing solution for 15 minutes.



3.

When blue color fades to clear, dentures are clean. It is safe to soak full dentures in solution overnight.

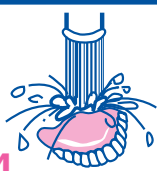

4.

Remove and rinse denture thoroughly in running water – they are now fresh and ready for use. Long-standing stains may need extra soaking.

- Recommended for use on full or partial dentures and removable orthodontic appliances
- For best results, use this product at least once a day.
- Full dentures may be soaked safely in this product overnight.

WARNINGS: Do not place tablets or denture cleaning solution in mouth. Do not use to clean natural teeth. See a dentist regularly. Keep this product out of reach of children. This product contains persulfates, which are known allergen. In case of accidental ingestion, seek professional assistance or contact the Poison Control Center immediately at 1-800-222-1222.

*Major Effervescent Denture Cleanser is not manufactured, distributed or licensed by the maker of Efferdent®

MAJOR 

EFFERVESCENT DENTURE CLEANSER

ANTI-BACTERIAL DENTURE CLEANSER

40 TABLETS

PN: 4322AGT
REV: 11/09

CONVENIENT RECLOSING TAB


EFFERVESCENT DENTURE CLEANSER

SEALED IN PRINTED PACKETS FOR YOUR PROTECTION.

MAJOR 

EFFERVESCENT DENTURE CLEANSER

ANTI-BACTERIAL DENTURE CLEANSER

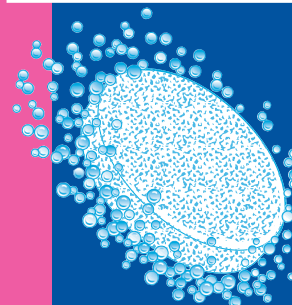

**KILLS MILLIONS
OF BACTERIA**

- Fights Odor
- Removes Stains

Compare to Efferdent®*

40 TABLETS

OPEN OTHER END

Distributed By
MAJOR PHARMACEUTICALS
31778 Enterprise Drive
Livonia, MI 48150 USA
M-97 Re-Order No. 479931
QUESTIONS ? (800) 616-2471



No Copy here Lot Numbering Area

MAJOR 

EFFERVESCENT DENTURE CLEANSER

ANTI-BACTERIAL DENTURE CLEANSER

40 TABLETS