



Micro-Scientific™ Stain-Away™ Instrument Wipes clean, polish and protect surgical instruments. They remove rust, stains, water deposits and tarnish, and polish dull, discolored metal.

- **Removes rust, stains, water deposits and tarnish.**
- **Rinses spot & residue-free.**
- **Convenient wipe is ready-to-use.**

Instrument Wipes

Order No.	Size	Physical Properties
IW20	20 Ct. Wipes	Form: Wipes Color: White Odor: Floral pH Concentration: 6.2

Uses	Temperature
<input type="checkbox"/> Presoak <input type="checkbox"/> Automatic Washer <input checked="" type="checkbox"/> Manual <input type="checkbox"/> Automatic Endoscope Reprocessor <input type="checkbox"/> Ultrasonic <input type="checkbox"/> Cart Washer	Usage: Room Temperature Storage: Cool, dry area

Product Features	Consumer Benefits
Removes Rust:	As well as stains, water deposits and tarnish.
Rinsing Agents:	Rinses spot & residue-free.
pH Near Neutral:	Product safe and compatible with all metals.
Bacteriostatic:	If stored properly, not favorable to bacterial growth.
User Friendly:	Convenient wipe is ready-to-use.

Procedures	Directions for Use
Directions:	<p>To remove stains, gently rub affected area with the wipe until stain is no longer visible. For heavily stained spots, rub area with greater pressure. Instruments must be washed after using wipes and before sterilization.</p> <p>Do not use on rings of gold- or black-handled instruments (tungsten carbide or SuperCut).</p>