

OPTIKLEEN®

Blade Cleaning System



Made in the U.S.A.



An efficient and safe method for cleaning tissue and other debris from the surfaces of scalpel blades prior to sterilization.



OPTIKLEEN®

Blade Cleaning System



DESCRIPTION: An efficient and safe method for cleaning tissue and other debris from the surfaces of scalpel blades prior to sterilization. Utilizing a proprietary soft, foamlike material, impregnated with OptiKleen instrument cleaning detergent that loosens proteins and other deposits, along with USP grade filtered rinse water, blades can be safely cleaned without scratching, dulling, chipping or breaking.

It's as easy as 1-2-3!
Simply clean, rinse and rinse again....



HOW IT WORKS: Insert the soiled blade into the soft scrubbing pad. Work the blade in a cutting-like fashion and allow the gentle scrubbing to clean and remove debris off of the blade surface and deposit it into the pad. Simply repeat in the two separate rinsing compartments and you're finished.

The results? A sparkling clean knife, ready for sterilization!
Lastly, at the end of the day, the tray is simply disposed of according to local regulations.



Product	Order#	Packed
12 individual disposable trays per dispenser box	OKD-001	2 box/case
Holding tray with cover	OKHT-001 I	each

Distributed By:

For technical data and product information contact:



MICRO-SCIENTIFIC®

THE LEADERS IN THE SCIENCE OF CLEANING & DISINFECTION

GURNEE, IL 60031
Telephone: 1-888-253-2536
Outside USA: 1-847-454-0835
www.micro-scientific.com