

# MICROLUBE™

## SUPER CONCENTRATE

### Instrument Lubricant



MicroLube™ Super Concentrate is specially formulated to be used as a lubricant and rust inhibitor for all types of surgical and dental instruments including carbon steel. It is compatible with all sterilization methods and if stored properly, can be reused for 28 days without becoming contaminated.

- **Reduces likelihood of rust, tarnish or corrosion.**
- **Extends life of instruments, resulting in fewer repairs.**
- **Patient-safe lubricant will not cause abscesses in body.**

# MicroLube™ Super Concentrate

## Instrument Lubricant



Order No.	Size	Physical Properties	
SP2.5C	2.5 Gallons (9.5 Liters)	<b>Form:</b>	Liquid
		<b>Color:</b>	White
		<b>Odor:</b>	None
		<b>pH Concentration:</b>	6.3
Uses		Temperature	
<input type="checkbox"/> Presoak	<input checked="" type="checkbox"/> Automatic Washer	<b>Usage:</b>	Room Temperature
<input type="checkbox"/> Manual	<input type="checkbox"/> Automatic Endoscope Reprocessor	<b>Storage:</b>	Cool, dry area
<input type="checkbox"/> Ultrasonic	<input type="checkbox"/> Cart Washer		
Product Features		Consumer Benefits	
<b>Rust Inhibitors:</b>		Reduces likelihood of rust, tarnish or corrosion. Extends life of instruments, resulting in fewer repairs.	
<b>Neutral pH:</b>		Product safe and compatible with all metals.	
<b>Non-silicone Lubricant:</b>		Patient-safe lubricant will not cause abscesses in body.	
<b>Bacteriostatic:</b>		If stored properly, not favorable to bacterial growth.	
<b>Environmentally-Friendly:</b>		Phosphate-free formula.	
Procedures		Directions for Use	
<b>Directions:</b>		Wear appropriate personal protective equipment based on safe work practices.	
<b>Automatic Washing Equipment:</b>		Add 1/20 fl. oz. (1.5 mL) to 1/4 fl. oz. (7.4 mL) per gallon (3.8 L) of water.	
		<b>Dilutions vary depending on temperature, water quality and equipment.</b>	