

SURFACE DISINFECTION - CAVIWIPES1™

IF GROSS DEBRIS IS PRESENT ON THE SURFACE:

CLEAN BEFORE DISINFECTING



1. Dispense CaviWipes1™ towelette.



2. Clean surface with towelette to remove debris and bioburden.



3. Discard used towelette.

DISINFECT AFTER CLEANING



4. Dispense another CaviWipes1 towelette.



5. Disinfect precleaned surface with towelette. Leave surface wet for one (1) minute.



6. Discard used towelette.

IF NO GROSS DEBRIS IS PRESENT ON THE SURFACE:

ONE STEP CLEANING AND DISINFECTING



1. Dispense CaviWipes1 towelette.



2. Disinfect surface with towelette. Leave surface wet for one (1) minute.



3. Discard used towelette.

PRODUCT INFORMATION

PART NO. US / Canada	PRODUCT DESCRIPTION	PACKAGING
13-5100 / 11-5100	CaviWipes1 (6" x 6.75"), 160 count	12 canisters/case
13-5150 / 11-5150	CaviWipes1 EXTRA LARGE (9" x 12"), 65 count	12 canisters/case
13-5155 / 11-5155	CaviWipes1 EXTRA LARGE Singles, (9" x 12"), 50 count	6 boxes/case
13-5224	CaviWipes1 Flat Pack (7" x 9"), 45 count	20 packs per case
13-1175	CaviWipes & CaviWipes1 Canister Wall Bracket	12 brackets/case

Note: Follow standard precautions for personal protection as appropriate.



CaviWipes1 Family
of surface disinfectants



For more information, visit metrex.com/education, or call 800.841.1428.

MKT-1055-1
77-1075-2