
# OMRON®

## INSTRUCTION MANUAL

### SELF TAKING BLOOD PRESSURE KIT

Model 0104

Model 0104MAJ

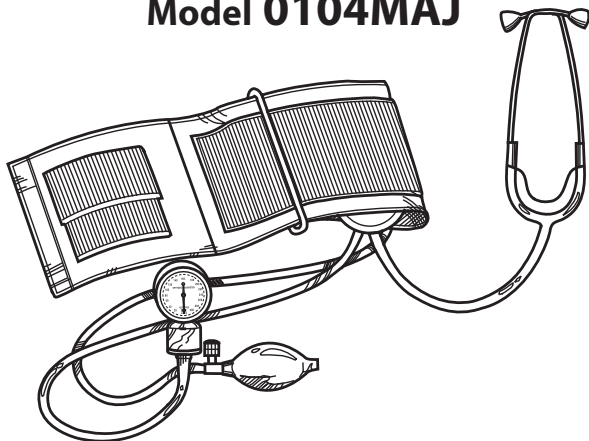ESPAÑOL

ENGLISH

# TABLE OF CONTENTS

Introduction .....	2
Important Safety Notes .....	3
Unit Assembly .....	4
Before Taking a Measurement .....	6
Taking a Measurement.....	7
Care and Maintenance .....	10
Limited Warranty .....	11
Blood Pressure Log .....	12

## INTRODUCTION

Thank you for purchasing the Omron Self Taking Blood Pressure Kit. The Omron Self Taking Blood Pressure Kit comes with the following components:

- Aneroid Gauge
- Standard Size Adult Arm Cuff (fits 9"-13" in circumferences) with attached Stethoscope (Chestpiece) for Model 0104
- Large Size Adult Arm Cuff (fits 13"-17" in circumferences) with attached Stethoscope (Chestpiece) for Model 0104MAJ
- Stethoscope Binaural
- Inflation Bulb and Valves
- English/Spanish Instruction Manual
- Storage Case

The Omron Self Taking Blood Pressure Kit is intended for home use.

## SAVE THESE INSTRUCTIONS

# IMPORTANT SAFETY NOTES

*To assure the correct use of the product, basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.*

## SAFETY SYMBOLS USED IN THIS INSTRUCTION MANUAL

 <b>WARNING</b>	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 <b>CAUTION</b>	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

-  Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.
-  **DO NOT** adjust medication based on measurement results from this blood pressure monitor. Take medication as prescribed by your physician. Only a physician is qualified to diagnose and treat High Blood Pressure.
-  The monitor is not intended to be a diagnostic device.
-  Read all of the information in the instruction manual and any other literature in the box before operating the unit.
-  Failure to assemble the parts correctly may result in injury or an incorrect reading.
-  Make sure the metal D-ring does not pinch your skin.
-  This device is intended for use in measuring blood pressure in the adult population.
-  Before you put the stethoscope binaural in your ears make sure the eartips are securely fastened to the binaural. Operate the device only as intended. Do not use the device for any other purpose.
-  Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
-  Changes or modification not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair.

# UNIT ASSEMBLY

## HOW TO FOLD THE CUFF

The cuff should already be folded through the metal D-ring when you remove the cuff from the box. If the cuff is not prefolded, lay the cuff flat in front of you. Place the metal D-ring on your left. The stethoscope faces up. Pull the end of the cuff through the metal D-ring. When the cuff is folded correctly, the hook and pile material faces the outside of the cuff loop.


## HOW TO ASSEMBLE THE UNIT

Lay the parts out as illustrated on the next page. Refer to the illustration to help you put the unit together.

Follow these four easy steps:

- ① Attach the gauge to tubing on the left.
- ② Attach the inflation bulb to the tubing on the right.
- ③ Attach the single end of the Y-tubing to the stethoscope chestpiece.
- ④ Attach the double end of the Y-tubing to the binaural.

# UNIT ASSEMBLY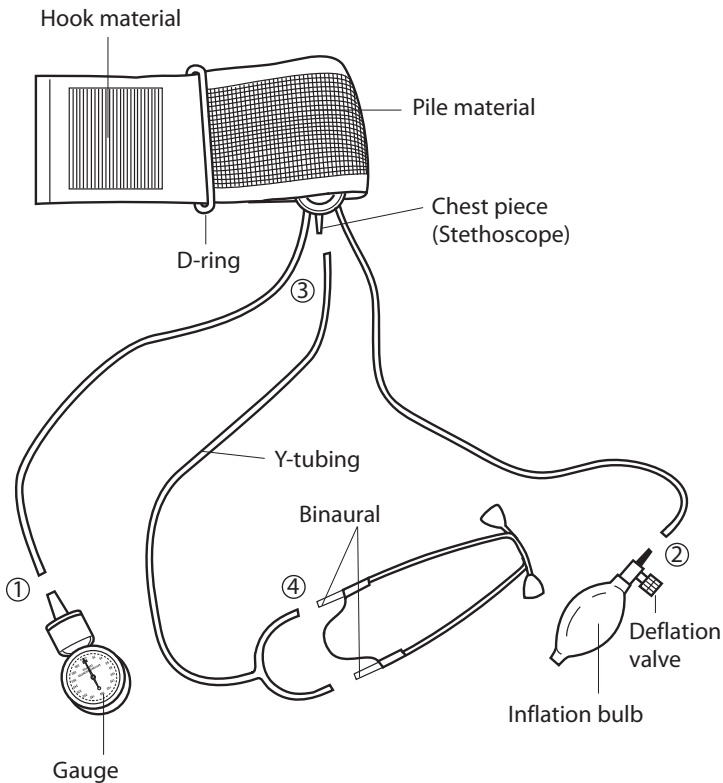

## **BEFORE TAKING A MEASUREMENT**


1. Avoid eating, drinking alcohol, smoking, exercising and bathing for 30 minutes before taking a measurement. Rest for at least 15 minutes before taking the measurement.
2. Stress raises blood pressure. Avoid taking measurement during stressful times.
3. Measurements should be taken in a quiet place.

# TAKING A MEASUREMENT


1. Make sure the parts are securely and correctly assembled.
2. Sit on a chair with your feet flat on the floor. Rest your left arm on a table so that the cuff is at the same level as your heart. Remain still and do not talk during the measurement.
3. Remove tight fitting clothing from your left arm.

**⚠ CAUTION:** Failure to assemble the parts correctly may result in injury or an incorrect reading.

4. Press two fingers one inch above the bend of your elbow on the inside of your arm. You may feel the pulse of the brachial artery. Even if you do not feel your pulse, the stethoscope will allow you to hear the pulse sounds.

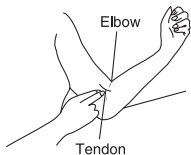
5. Slip the cuff up your arm. Place the bottom edge of the cuff one-half inch above the elbow. Adjust the cuff so the chestpiece sits on the inside of the arm over your brachial artery.
6. Pull the end of the cuff downward until the cuff fits snugly.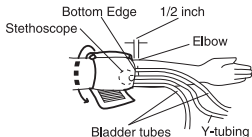


**⚠ CAUTION:** Make sure the metal D-ring does not pinch your skin.

Press the hook material firmly against the pile material. Make sure the chestpiece sits over the brachial artery.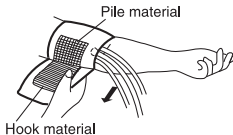

## TAKING A MEASUREMENT

7. Place your arm on a table so the cuff is at the same level as your heart.

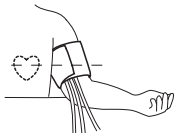**⚠ CAUTION:** Before you put the stethoscope binaural in your ears make sure the eartips are securely fastened to the binaural.

8. Insert the eartips into your ears. Adjust the metal binaural for comfort.

9. Clip the gauge to a hard cover book on the table in front of you. Hold the inflation bulb in your right hand. Close the air deflation valve attached to the inflation bulb by turning the valve clockwise.

10. To inflate the cuff, slowly and steadily pump the inflation bulb.

**NOTE:** If you know your systolic reading, pump the inflation bulb until the needle on the gauge reaches approximately 50 mmHg above your average systolic reading. If you do not know your systolic reading, pump the inflation bulb until the needle on the gauge reaches 180 mmHg.


**⚠ CAUTION: DO NOT** inflate the cuff above 280 mmHg. You may damage the unit or cause personal injury. If you are unsure, consult your healthcare provider for the correct inflation level.

## TAKING A MEASUREMENT

11. Slowly open the air deflation valve by turning it counterclockwise. Release the air at a rate of 2 mmHg per second.

12. Listen carefully for your pulse beat. Watch the gauge needle. When you hear the first thumping sound note the reading on the gauge. This is your **SYSTOLIC** blood pressure value.

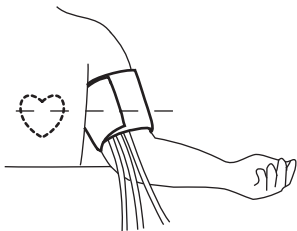13. When you no longer hear any sounds note the reading on the gauge. This is your **DIASTOLIC** blood pressure value.
14. Turn the air deflation valve counterclockwise to release the remainder of the air. Remove the stethoscope binaural from your ears. Remove the cuff from your arm. Record your reading in the Blood Pressure Log.

**NOTE:** If you make a mistake, do not immediately reinflate the cuff. Release all the air in the cuff. Wait 5 minutes. The wait time allows the arteries to return to the condition prior to taking the blood pressure measurement.

## CARE AND MAINTENANCE

Your new blood pressure kit has been carefully checked to ensure reliability and accuracy prior to shipment and use. As with any sensitive instrument subjected to repeated use, the manufacturer recommends your blood pressure gauge be checked annually by a medical professional for proper calibration. The unit is properly calibrated when the needle rests at zero.

**To keep your blood pressure kit in the best condition and protect the unit from damage follow the directions listed below:**

Avoid washing or moistening the cuff. Never immerse any of the parts in water.

Store in a safe and dry location. Do not fold the cuff and tubing tightly.

Avoid subjecting the instrument to strong shocks, such as dropping the gauge on the floor or hitting the surface of the stethoscope. When the cuff is fully deflated, the needle stays within the zero indicator on the gauge. If the needle points outside of the zero indicator the gauge gives inaccurate readings. Return the unit to the manufacturer for calibration.

Use the unit consistent with the instruction provided in this manual.

Use only authorized parts and accessories.

# LIMITED WARRANTY

Your Omron Self Taking Blood Pressure Kit is warranted to be free from defects in materials and workmanship appearing within 1 year from the date of purchase, when used in accordance with the instructions provided with the kit. The above warranties extend only to the original retail purchaser.

We will, at our option, repair or replace without charge any components of the Blood Pressure Kit covered by the above warranties. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service contact Customer Service by calling **1-800-634-4350** for the address of the repair location and the return shipping and handling fee.

Enclose the Proof of Purchase. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

**THE FOREGOING IS THE SOLE WARRANTY PROVIDED BY OMRON IN CONNECTION WITH THIS PRODUCT, AND OMRON HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IMPLIED WARRANTIES AND OTHER TERMS THAT MAY BE IMPOSED BY LAW, IF ANY, ARE LIMITED IN DURATION TO THE PERIOD OF THE ABOVE EXPRESS WARRANTY.**

**OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER SPECIAL, INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.**

This warranty provides you with specific legal rights, and you may have other rights that vary by jurisdiction. Because of special local requirements, some of the above limitations and exclusions may not apply to you.

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Copyright © 2010 Omron Healthcare, Inc.
**Omron Healthcare, Inc.**