**ENGLISH:**
Instructions for use

Autolet Impression is an advanced lancing device, which provides maximum comfort and control when obtaining a blood sample.

Containing unique, patented Comfort Zone Technology®, Autolet Impression increases the comfort of lancing by masking the sensation of pain to provide a more comfortable blood sample.

Please read the instructions carefully when using Autolet Impression for the first time and retain for future reference.

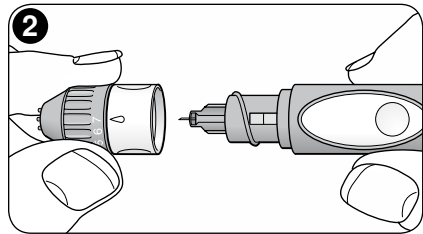
Wash the sampling site with warm water. Rinse and dry thoroughly.

STEP 1:
Fully insert a new lancet into the lancet holder. Gently twist and pull off the protective cap (fig. 1).

STEP 2:
Screw the depth adjuster onto the device (fig. 2).

STEP 3:
Set the penetration depth by turning the depth adjuster ('1' = minimum and '7' = maximum depth). It is recommended you start at the minimum depth (fig. 3).

STEP 4:
Prime Autolet Impression by holding the main body and pulling the cocking mechanism until you hear a click (fig. 4).

STEP 5:
Set the spring force by turning the force adjustment ring until the marker on the device body is in line with the desired setting (fig. 5).

□ 1 = Lightest □ 2 = Normal □ 3 = Strongest

N.B. Adjusting the force can be used as an alternative to increasing the penetration depth.

STEP 6:
Hold Autolet Impression firmly against the finger allowing the unique Comfort Zone Technology to activate. Press the release button. If required, gently massage the finger from the base to the fingertip (fig. 6).

STEP 7:
Remove the depth adjuster and pull the used lancet out. Discard the used lancet carefully in a suitable sharps container (fig. 7).

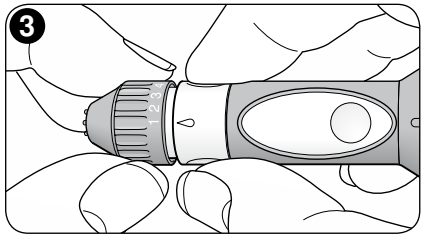
ALTERNATE SITE TESTING (AST)

Choose a penetration site on a soft, fleshy area of your arm, or other recommended site, away from bone, and free of visible veins and hair.

Obtaining a sample using the Alternate Site end cap.

- Follow Step 1 as shown above.
- Replace the normal depth adjuster with the clear Alternate Site end cap (fig. A).
- Follow steps 4 and 5 as shown above.
- Rub the test site for a few seconds until it is warm.

Hold Autolet Impression firmly against the site for a few seconds. Press the release button (fig. B). Continue to hold Autolet Impression against the site and gradually increase the pressure for a few seconds. Allow a sufficient amount of blood to form under the cap.

Lift Autolet Impression straight up being careful not to smear the blood sample (fig. C).

CLEANING

Clean with a cloth and warm water as required. Do not immerse in water.

SAFETY INFORMATION

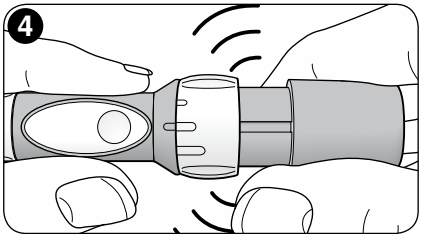
Autolet Impression is intended for self-monitoring by a single user only and must not be used in multi-patient clinical environments.

Do not use if cap has been previously removed. Due to the risk of cross infection, this single use device is for self-monitoring by a single user only. Reuse of this single use device will affect safety, performance and effectiveness, and exposes users to unnecessary risks.

WARRANTY INFORMATION

The warranty offered by Owen Mumford Limited ("we") in relation to this product is set out at owenmumford.com. PLEASE READ THIS WARRANTY IN FULL. Your rights under the laws of your country are not affected.

 For date of manufacture please see dispenser box.

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Unilet ComforTouch
23G 28G 30G

CE
0120
STERILE

Do not use if cap has been previously removed. Due to the risk of cross infection, this single use device is for self-monitoring by a single user only. Reuse of this single use device will affect safety, performance and effectiveness, and exposes users to unnecessary risks. Please note that not all products are available in all countries.

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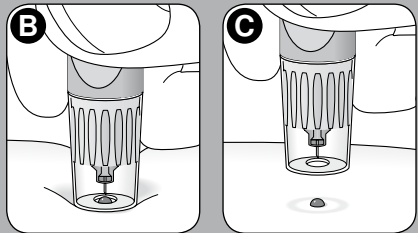
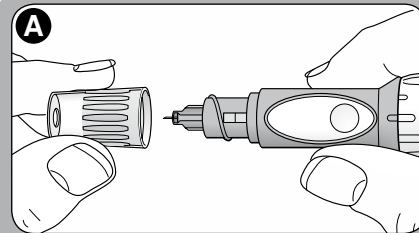
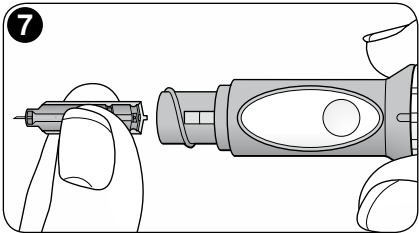
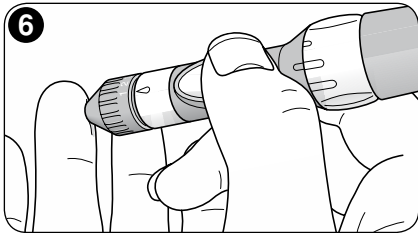
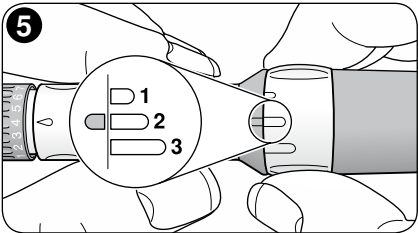
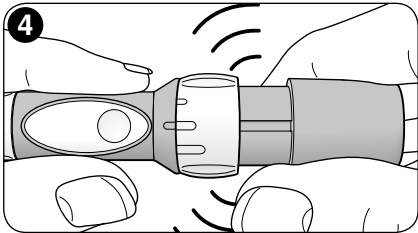
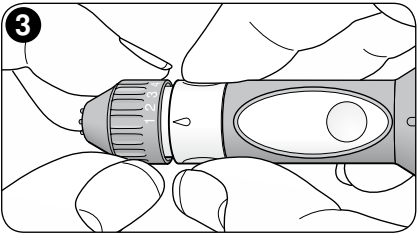
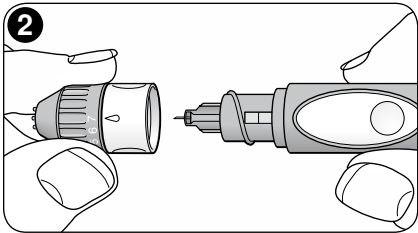
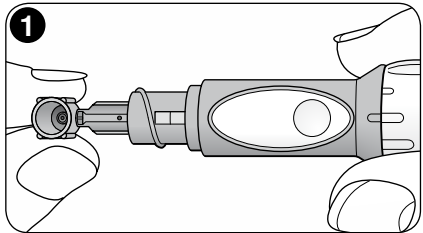
Autolet® Impression and Unilet® ComforTouch™ made in England for Owen Mumford.
Unilet® Eco/Excelite made in China, packaged by Owen Mumford.

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Autolet®, Unilet® and Comfort Zone Technology® are registered trademarks of Owen Mumford Ltd. International Patents Pending.
Owen Mumford GmbH.
Medical Device Company Ltd