

How to use **Autolet® impression**


Simple and adjustable



Insert a new lancet, remove the protective cap and attach the depth adjuster.



Adjust the penetration depth, selecting one of the seven settings.

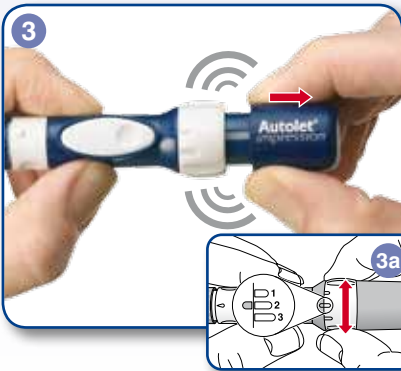

Prime the device by pulling the main body and the device sleeve apart until you hear a **CLICK**, then choose your desired force setting.



Hold the device firmly against the finger, press the release button. Remove the depth adjuster and discard the used lancet in a suitable sharps container.

Capillary Blood Sampling

Autolet® Impression



HEAD OFFICE
Owen Mumford Ltd
Brook Hill, Woodstock
Oxfordshire OX20 1TU, United Kingdom
T : +44 (0)1993 812021
F : +44 (0)1993 813466
E : info@owenmumford.co.uk

UK / IRELAND SALES & MARKETING
Owen Mumford Ltd
Primsdown Industrial Estate
Worcester Road, Chipping Norton
Oxfordshire OX7 5XP, United Kingdom
T : +44 (0)1608 645555
F : +44 (0)1608 646546
E : info@owenmumford.co.uk

THE AMERICAS
T : +1 800-421-6936
F : +1 770-977-2866
E : info@owenmumfordinc.com

FRANCE
T : +33 (0)2 32 51 88 70
F : +33 (0)2 32 21 96 95
E : contact@owenmumford-fr.com

GERMANY
T : +49 (0)6026/97750
F : +49 (0)6026/977529
E : info@owen-mumford.de

CHINA
T : +86 (0)21 6032 3528
F : +86 (0)21 6032 3782
E : info@owenmumford.co.uk

owenmumford.com

- OUR ACCREDITATIONS INCLUDE:**
- ISO13485 Certification
 - Medical Device Directive 93/42/EEC
 - CMDCAS 13485 (Canadian Health Authority)
 - Environmental Management Systems ISO14001



AT/0290-LT/MCC0115/8

Comfort and control



Autolet® impression

Lancing device



More reason to smile

owenmumford.com

Autolet[®] impression

Lancing device


Autolet[®] Impression is an advanced lancing device designed to provide maximum comfort and control when obtaining a blood sample, encouraging regular glucose testing and management of blood sugar levels.

Comfort Zone Technology[®].

When pressed against the sample site, a series of 8 raised dots on the platform stimulate the nerve endings. This sends a powerful message of comfort to the brain – masking the weaker pain stimulus from the lancet.

The Gate Theory

Comfort Zone Technology[®] is based on the well-known ‘Gate Theory’¹, and works by stimulating the nerve endings in the fingertips to produce a comfortable sensation which masks the pain sensation from the lancet.

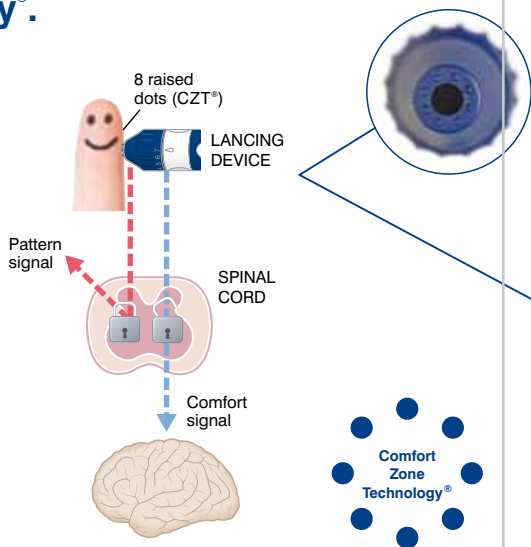

Precise adjustment for a personal experience.

To achieve optimal blood flow and comfort, **Autolet[®] Impression** offers 21 settings, allowing the user to select the combination which delivers the correct blood volume first time. Seven depth settings and three force settings allow exact adjustment for all skin types and sampling requirements.



Large button and ergonomic design make Autolet[®] Impression easy to use.

Many people with diabetes also have dexterity issues, making blood sampling difficult. The ergonomic design of **Autolet[®] Impression** has been developed to make it simple to use.




Alternate site sampling

Autolet[®] Impression comes with a clear alternate site end cap that allows the device to be used for testing on alternate sites such as the forearm.



Clinical Evidence²

Lancet trials showed that **Comfort Zone Technology[®]** (CZT[®]) reduced the overall pain experienced during the blood sampling procedure when compared to devices without **Comfort Zone Technology[®]**.

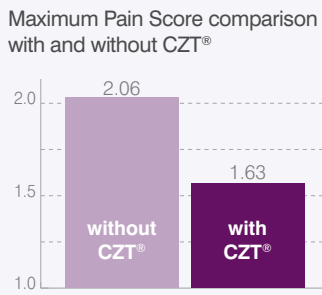

1. Melzack R, Wall PD. The challenge of pain. Penguin, Harmondsworth. 1982 p233 2. Dewland P, Edwards C. A single-blind, randomised, 8-way crossover study to compare the blood volume and pain perception of capillary blood sampling – March 2007. † Except Roche Multiclix and Softclix Plus, LifeScan OneTouch Delica.

Compatibility

Unilet[®] lancets fit **Autolet[®] Impression** and are compatible with most capillary blood lancing devices[†].

Lancet	Gauge	Description	
Unilet [®] ComforTouch [™]	28G	Thin recappable lancet	Box of 100: AT0465
Unilet [®] Excelite	28G	Confident and comfortable testing	Box of 100: AT0535
Unilet [®] Excelite	33G	Confident and comfortable testing	Box of 100: AT9585
Unilet [®] Eco	28G	Confident and comfortable testing	Box of 100: AT0935

