Instructions for use
Mode d'emploi
Instrucciones de uso

Made in England by

OWEN MUMFORD
Making a World of Difference

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
Select a slot which fits firmly around the base of your flaccid (non erect) penis (closest to your body). Try the largest size first.

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

8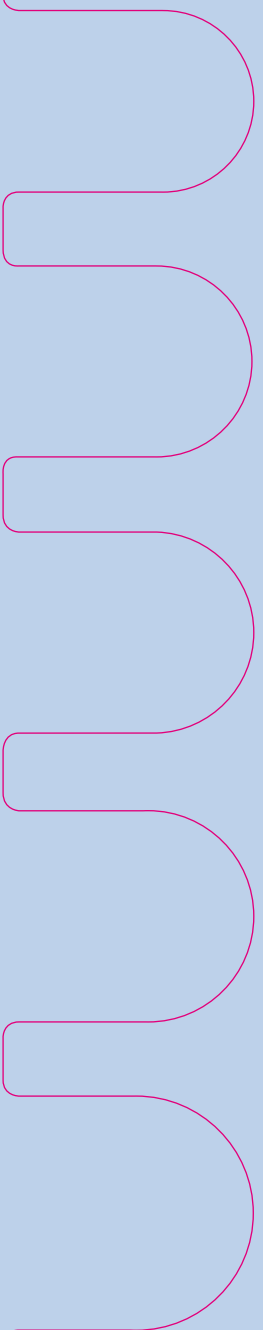

Contents / Contenu / Contenidos

2	What is Impotence?
2	What causes Impotence?
2-3	How can I manage Impotence?
3	Using Vacuum Devices to treat Impotence
4	Some Special Points to Remember
5	Safety Precautions
5	Risks
6	Warnings
7-8	How the Rapport Classic Vacuum Therapy Device (V.T.D.) Works
9	Safety Features
9	Choosing the Correct Constriction Ring
9-10	Choosing a Transfer Sleeve
10	Assembling the Pump and the Penile Tube
10	Mounting the Constriction Ring
11-12	Using the Pump to Create an Erection
13	Cleaning Instructions and Care of your Rapport Classic V.T.D.
14-15	Troubleshooting Guide
16	Glossary of Terms
17	Warranty

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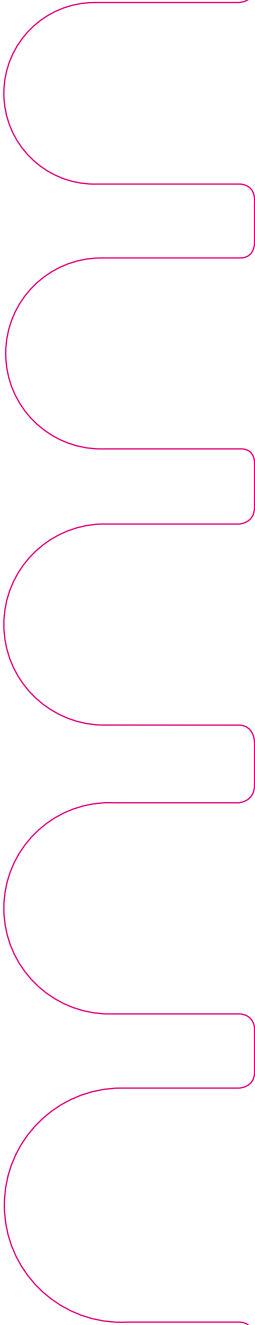What is Impotence?

Impotence (more accurately called erectile dysfunction) is the consistent inability to gain and maintain an erection of sufficient firmness that will allow penetration during sexual intercourse.

Erections happen when the brain transmits a sexual signal to the blood vessels in the penis; the vessels then dilate, drawing blood into the penis to create a firm erection.

Erectile dysfunction can occur if the nerves and blood vessels are not functioning properly.

What causes Impotence?

Impotence can happen gradually over many months or it can suddenly occur as a problem in certain sexual situations. The causes can be both psychological and physical; but the physical causes often have a psychological element as well.

One cause commonly blamed is age. The aging process may affect the frequency and intensity of desire, but does not normally affect the ability to produce an erection. As we grow older, however, we are more likely to suffer from medical conditions that will affect the physical erection process.

Diabetes affects both the nerve and blood supply to the penis. 50% of diabetic men will experience erectile problems at some point in their life.

The most common blood supply problems are arteriosclerosis which slows the blood supply to the penis, or a venous leakage problem where blood drains away too quickly from the penis causing a loss of erection.

Certain medications used to treat high blood pressure or depression can lead to impotence, as can overindulgence in alcohol.

Nerve damage occurs with multiple sclerosis, spinal cord injuries or after some types of prostate surgery.

The stress and strain of everyday life, severe marital problems or the loss of employment can cause impotence and, since impotence itself is a source of anxiety, the problem can be self-reinforcing.

How can I manage Impotence?

The first thing to do is admit to yourself that you have a problem and perhaps discuss it with your partner.

Visiting your physician to have some simple test performed to determine whether your impotence has a physical or psychological cause is a good first step.

There are several treatment options to manage impotence:

- Side effects of some common drugs used to treat other conditions can cause impotence i.e. beta blockers, antidepressants. A simple change in prescription is all that is required.
- Oral medication is suitable for some patients who do not suffer from heart disease.
- Self-injection of a blood vessel dilator into the penis each time before intercourse is a widely accepted treatment.
- Implanted prostheses are highly successful but require an operation under anesthetic.

- If surgery and injections or other medicinal therapies are not favorable, the vacuum method offers a simple, reliable treatment for impotence.
 - It consists of a tube that is placed over the penis to create a vacuum. This vacuum causes blood to be drawn into the penis and the blood is then retained by a constriction ring which is rolled onto the base of the penis and worn during intercourse.
 - The method is easy to learn in the privacy of your home and can be a normal part of the lovemaking process.
 - Vacuum therapy has a very good success rate in the treatment of impotence.

Conditions where the use of the Rapport Classic Vacuum Therapy Device (V.T.D.) may need special assistance:

- Blindness
- Peyronies disease
- Spinal cord injuries
- Multiple Sclerosis
- Amputation

Conditions where the Rapport Classic V.T.D. should NOT be used:

- Men on anticoagulant therapy
- After removal of a penile implant
- Fibrosis caused by penile injections
- History of sickle cell disease or any other bleeding disorder
- Pelvic trauma
- History of Priapism
- Any problem where sexual activity may endanger health

Using Vacuum Devices to treat Impotence

Realistic expectations, high motivation (including that of your partner), libido, manual dexterity, and a willingness to vary the couple's sexual techniques to incorporate the device are all required to bring about a welcome result. Although the male orgasm may not be affected, ejaculation may be blocked or retrograde may occur as a result of using vacuum devices.

You are encouraged at first to have practice sessions without use of the constriction rings for about 15 minutes a day. This should be continued once a day for at least a week or until a satisfactory erection is achieved and any initial discomfort subsides.

Beginning with the loosest constriction ring and gradually increasing to tighter ones, you should select one that maintains the erection, but is not painful. The ring should not be kept on for any period longer than 30 minutes.

Liberal use of a water soluble lubricant is important to avoid penile tissue adherence to the penile tube and to establish an airtight seal around the penile tube. The lubricant should also be applied to the area where the penile tube and body meet.

Some Special Points to Remember

We recommend that the first few times you use the Rapport Classic V.T.D. you do not make your penis any harder than necessary. Your penis may not be used to the amount of pressure, size and hardness you will experience. Increase the firmness of your erection gradually.

If you experience any pain when you use the vacuum pump, lower the vacuum pressure immediately by pressing the release valve.

Be sure to start by using a large size constriction ring to ensure that the constriction on the penis is no more than necessary.

If your erection does not stay firm and hard, the constriction ring may be too large and not enough blood is staying in the penis. Use the next size smaller.

If you feel pain with the constriction ring on your penis you may be using a ring that is too small. Use the next size larger.

If the smallest constriction ring does not maintain your erection, try using two rings together. You can wear any two rings together if it helps to maintain your erection.

To prevent any tangling of the constriction ring in the pubic hair, you can use water soluble lubricating jelly in the hair before placing the ring on the base of your penis.

Alternatively, you may wish to remove the pubic hair for ½ - 1 inch around the base of the penis to prevent any painful tangling.

Always position the constriction ring with the handles to the top and bottom while on the base of your penis. This will help you with ejaculation and easier removal.

With the constriction ring in place your penis may have a bluish color and the head may appear much larger than normal. Do not be alarmed, this is normal when your penis is filled with blood.

While wearing the constriction ring, ejaculation does not necessarily occur as heavily as normal unless you are able to pull the bottom constriction ring handle down and away from your penis to relieve the pressure on the urethra.

Your penis will stay erect until the constriction ring is removed. This **MUST** be done within 30 minutes.

You can obtain a second erection, but you must wait at least 60 minutes for the blood supply to return fully to your penis. You may then repeat the process of creating an erection.

If you have difficulties creating and maintaining an erection try putting the constriction ring onto the base of your penis first, making sure that you tuck the ring inside the penile tube and then draw the erection with the pump through the ring. This is effective but the resulting erection may be slightly smaller than normal.

Safety Precautions for the use of the Constriction Rings

- 20 minutes is the recommended time limit for wearing the constriction rings. Never wear the rings more than 30 minutes as this may cause strangulation and subsequent gangrene of the penis.
- Do not risk falling asleep while wearing the constriction ring. You may damage the penis, as stated above.
- Do not wear the constriction ring while under the influence of alcohol and drugs. You may forget to remove the ring and cause permanent damage to your penis.
- Allow a minimum of 60 minutes between uses.
- Frequent use of the constriction rings in a short period of time may result in severe bruising at the base of your penis.
- Use the largest constriction ring that holds the erection.
- The constriction rings should be removed immediately if there is severe sudden pain, a sudden change in the color of your penis or bruising occurs.
- If any problems occur, discontinue the use of the constriction rings and consult your physician.
- Wear the constriction ring only while engaged in intercourse.
- Owen Mumford is not liable for any misuse of this product due to failure to read or understand these instructions.
- Penile constriction rings are not intended for use as a contraceptive / birth control.

Risks

The most common risks experienced are:

Bruising of the penis. This often happens when you first use the device. Bruising can range from small red spots on the skin of your penis to painful black or blue bruises. This is best avoided by increasing the amount of vacuum pressure slowly when first using the device.

Pain in the penis. This can occur when the vacuum pressure is applied. If this does happen, stop pumping immediately and release the pressure using the pressure release valve. If the pain continues, consult your physician before using the device again.

A hematoma may form. This is a severe form of bruising caused by bleeding under the skin. Do not use this device again until all signs of bruising have disappeared. If it happens repeatedly, stop using the device and consult your physician.

Wrong use of the constriction rings. The rings are safe if used properly, but if they are left on your penis for more than the recommended 20-30 minutes, they can cause permanent damage to your penis by restricting the blood supply.

Permanent penile injury. This can only happen if the pump is used incorrectly and causes swelling of the penis or by making pre-existing conditions worse. If the instructions are followed correctly this should not happen.

Warnings

Cautionary points on the use of vacuum therapy devices.

If you have symptoms of erectile dysfunction, (i.e. inability to achieve an erection that is sufficient for sexual intercourse), consult your physician prior to using this device to avoid a potentially harmful delay in diagnosing any of the most common causes of this condition, such as diabetes, multiple sclerosis, cirrhosis of the liver, chronic renal failure or alcoholism.

- Use the minimum amount of vacuum necessary to achieve an erection. This will prevent bruising.
- Stop drawing vacuum if pain occurs. Release the pressure with the pressure release valve.
- Caution is advised when using a vacuum pump following removal of a penile implant because of possible surgical scarring and subsequent impairment.
- Misuse of a vacuum pump may aggravate already existing medical conditions such as Peyronie's disease, priapism and urethral stricture.
- Misuse of the vacuum pump could result in swelling of the penis and / or permanent injury.
- Use of the vacuum pump may bruise or cause rupture of the blood vessels immediately below the surface of the skin or within the deep structures of the penis or scrotum, which may produce hemorrhage or hematoma.
- If any problems occur, discontinue the use of the vacuum pump and consult your physician.
- Do not use the vacuum pump while under the influence of alcohol or drugs.

How the Rapport Classic V.T.D. Works

An erection is created by means of a vacuum, which draws blood into the penis. The erection is maintained by placing a constriction ring at the base of the penis, which ensures that the blood remains in the penis until the ring is removed.

The constriction ring should be a snug, comfortable fit without being too tight.

The use of the Rapport Classic V.T.D. is not intended to replace your doctor's diagnosis of the underlying cause of your impotence, but to help achieve and maintain an erection. It is always best to discuss your individual situation with a doctor when using this treatment.

“ Studies have shown that by the use of vacuum devices, in excess of 80% of men are able to achieve a degree of penile tumescence sufficient to allow completion of sexual intercourse. ”

The Journal of Sexual Health June 1993 Volume 3 Number 6

Initially it may take some practice to familiarize yourself with the Rapport Classic V.T.D., so please take the time to read this instruction book, and to practice the technique of creating and maintain your erection, before you attempt sexual intercourse.

A demonstration DVD is included. Watching the DVD, particularly with your partner, may help you achieve success more quickly. The DVD shows a live model demonstrating the Rapport Classic V.T.D. but is not intended to be a substitute for reading the following instructions. This may be offensive to some people.


Practice Really Does Make Perfect

If you have not had an erection for some time, you may not be able to create one initially. It may take several practice sessions to achieve an erection for the first time.


If you have not had a firm usable erection for more than one year, you may need additional practice.

The Rapport Classic V.T.D. is a complete system with everything you need for immediate use.

Inside the discreet storage bag you will find: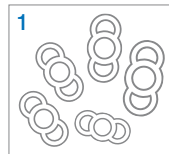

1 Five sizes of Rapport constriction rings. Five numbered rings vary in size. Number 4 is the smallest ring and Number 8 is the largest. The center ring is designed to encircle the base of the penis. The outside rings serve as handles for easy application and removal.

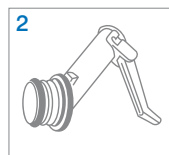
2 An easy to use Vacuum Pump which has a specially angled handle to make creating the erection simple.

As a safety measure the vacuum pressure is controlled by a valve that will not allow the pressure to raise above 360mm of Hg. The vacuum pressure can be released at any time by pressing the pressure release valve on the top of the pump.


3 A loading cone to assist the mounting of the constriction rings onto the transfer sleeve.

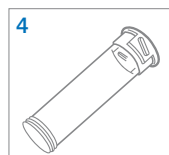
4 A penile tube in durable clear material, which attaches to the pump unit at one end and the transfer sleeve with the constriction ring at the other.


5 Two transfer sleeves, which give a choice of internal diameter. Both are specially shaped to hold the constriction ring and make it easy to transfer the ring onto the base of the penis.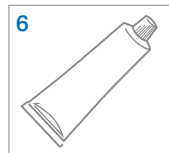

6 A tube of water soluble lubricating jelly. This helps to create the vacuum seal and prevent the penis from adhering to the penile tube.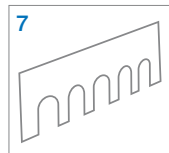

7 A constriction ring sizing guide for easy selection of the correct size ring and a warranty card. These can be found at the back of this instruction book.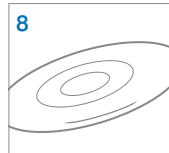

8 An instruction DVD, which shows how to put the Rapport Classic V.T.D. together and demonstrates how to create an erection. A live model is used in the DVD which may be offensive to some people.

Safety Features of the Device

Special safety features have been included into the Rapport Classic V.T.D. to make it as safe to use as possible.

The vacuum pump. This has been designed so that it cannot reach a dangerously high pressure no matter how hard you press the pump. Any excess pressure is automatically released through the pressure release valve. This helps to prevent bruising of the penis particularly when first using the Rapport Classic V.T.D.

The pressure release valve. This allows the vacuum pressure in the penile tube to be immediately released when the valve knob is pressed down. The valve knob is situated near the pump handle and should be pressed at any time when pain is felt in your penis as you are creating the erection.

The penile tube. This is made from clear plastic so that it is possible to see your erection developing inside the tube. This helps to give you confidence in the working of the vacuum pump.

The transfer sleeves. These are made from a soft material for comfort against your skin and are specially shaped to allow the constriction ring to slide down onto the base of the penis with as little discomfort as possible.

The constriction rings. These are made of a soft material for comfort and ease of use. The two outer handles make the ring easy to move when loading it onto the transfer sleeve and removing from the penis after intercourse. They can be used for the speedy removal of the ring from the penis if this is necessary at any time.

Choosing the Correct Constriction Ring

One of the most important parts of the Rapport Classic V.T.D. is the constriction ring. The ring must encircle the penis and fit comfortably tight.

The constriction ring works by retaining the blood in the penis to maintain a firm erection.

Using the constriction ring sizing guide provided, place your flaccid (non erect) penis into the slots until you find one which fits the circumference of your penis. Each slot has a ring size marked above it which corresponds to the ring you require.

If the Rapport Classic V.T.D. gives you a good erection, and with the constriction ring on your penis, your erection becomes soft within five minutes or less, you will need to try a ring one size smaller.

If the constriction ring seems too tight, next time use one size larger to see if it will maintain your erection. Try different sizes until you are satisfied with their fit. There are 5 sizes to choose from.

Choosing a Transfer Sleeve

There are two transfer sleeves provided with two internal diameters. Depending on the size of your penis you should choose which of the two is likely to be the most suitable. Remember that your erect penis will need to be withdrawn through the transfer sleeve.

The transfer sleeves are specially shaped so that the constriction ring can be easily loaded and held in the flat groove.

The unique shape also allows easy passage of the constriction ring off the transfer sleeve and onto the base of your penis.

Because the transfer sleeves are made of a soft material they help to make a good seal against your body to achieve the vacuum inside the penile tube which creates the erection.

Assembling the Pump and the Penile Tube

The pump head has two small pegs on each side above the O-ring; these should be positioned into the two grooves on the top of the penile tube.

The two parts are then locked together with a twist so that the two pegs slot down and locate at the lower end of the groove. (Fig.1)

If these two parts are then locked together correctly it will ensure the unit is sealed tight enough to create a vacuum inside the penile tube. (Fig.2)

Mounting the Constriction Ring

When you have selected the most suitable transfer sleeve, place it firmly onto the clear end of the penile tube. (Fig.3)

Fit the loading cone onto the transfer sleeve and apply a little water soluble lubricating jelly to the top of the cone.

Holding the constriction ring by outer handles, simply stretch and pull the ring down the loading cone until it slips onto the transfer sleeve.

Make sure that the constriction ring is located into the flat groove at the center of the transfer sleeve. (Fig.4)

The constriction ring is now mounted and ready for use.

Before using the Rapport Classic V.T.D. make sure you use the water soluble lubricating jelly to lubricate the end of the transfer sleeve. Also lightly lubricate your penis to prevent it from sticking to the inside of the penile tube during insertion or while using the pump. (Fig.5)

Using the Pump to Create an Erection

- 1 To ensure a vacuum will be created, insert your penis into the penile tube and apply slight pressure to seal the transfer sleeve against your body at the base of the penis. (Fig.6)

Make sure the constriction ring on the transfer sleeve is in the correct position with the handles to the top and bottom.

To prevent the penile tube from fogging, submerge **the tube only** in warm water (**never** immerse the vacuum pump itself in water). Again, make sure the handles of the constriction ring are positioned to the top and bottom of the penis. This will be more comfortable in use and allow easier ejaculation.

Steady the tube with one hand to prevent any jerky movements and squeeze and release the pump handle with the other hand. This works the pump to create the necessary vacuum. (Fig.7)



You may find that creating the erection will be easier and more comfortable if you stand up to use the pump. You can always steady the tube by placing it on a flat surface such as a table.

If you are unsteady when standing up, create the erection sitting down on the edge of a chair.

- 2 Continue pumping slowly to create the erection. If you feel any pain, stop pumping and press the pressure release button on the pump to eliminate the vacuum. Take your penis out of the tube, then reinsert it and start pumping again slowly. Repeat this if necessary. Pumping slowly is most beneficial, as this draws the blood into your penis at the correct rate.

Please take care not to pump too fast as this may cause bruising.

NOTE: For your safety, the vacuum pump is unable to reach a dangerous pressure. The vacuum is automatically released through the release valve when pressure reaches 360mm of Hg.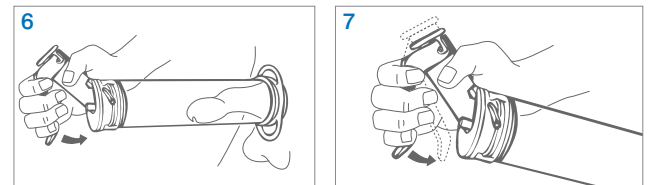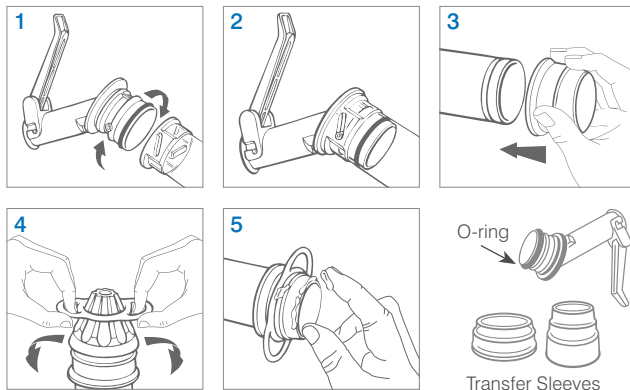

- 3** When the desired erection has been achieved, continue to hold the tube tightly against your body. Hold the tube with one hand, still pushing it against your body and with the other hand, ease the constriction ring off the transfer sleeve and onto the base of your penis. Always position the constriction ring with the handles to the top and bottom while on the base of your penis. This will help with ejaculation and easier removal. (Fig.8)

Because of the special shape of the transfer sleeve, when the constriction ring is eased out of the flat groove it will then slip easily off the transfer sleeve and onto the base of your penis.

Some vacuum pressure may be lost during this transfer, but you will still need to press the release valve to lower the vacuum pressure and enable you to remove the penile tube. (Fig.9)

If you find that placing the constriction ring onto your penis is difficult, try releasing a little of the vacuum by pushing the penile tube away from your body slightly then rolling the ring off the transfer sleeve.

- 4** With a constriction ring in place, a lasting erection should now be maintained and sexual intercourse can be accomplished.

NOTE: If a condom is to be worn, it should be put on as a last step. Do not wear a condom during the vacuum process. (Fig.10)

- 5** To remove the constriction ring after intercourse, simply grasp the outer handles to stretch the central ring. As the blood is released from the penis it will decrease in size and the ring may be easily removed.

Cleaning Instructions and Care of your Rapport Classic V.T.D.

1 The Constriction Rings

After each use, the constriction ring should be washed with mild soap in warm water, rinsed well and dried thoroughly. Return the ring to its bag.

Make sure you only use water soluble lubricating jelly with the constriction rings. Your rings should last longer if cared for properly.


2 The Penile Tube and Transfer Sleeve

After each use, the penile tube and transfer sleeve should be washed thoroughly with a mild soap in warm water. Rinse and dry well before replacing them into the carrying bag.

3 The Vacuum Pump





Very little maintenance is needed for the vacuum pump, but we recommend that you occasionally wipe the vacuum pump with a damp cloth to remove any dirt or lubrication. **Do not immerse the vacuum pump in water, as this may cause handle to stick and it will not be possible to create a vacuum.**

Please take care when wiping the underside of your vacuum pump as the non return valve (3 pronged rubber valve) can be pushed into the piston chamber with only a slight force. **If this valve is not in position, no vacuum will be achieved.**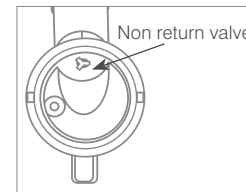

It will be necessary from time to time to lubricate the O-ring on the pump head, particularly if you are finding it difficult to attach or remove the pump head to and from the penile tube.

After cleaning the pump head apply a little petroleum jelly to the O-ring (the coloured band) making sure you smear the lubricant all the way around the O-ring. This will ensure a total seal between the pump head and the penile tube and maintain the vacuum inside the penile tube.

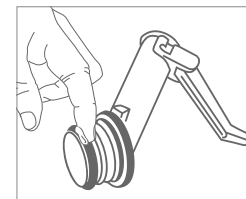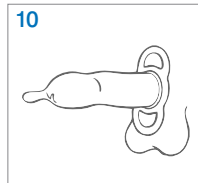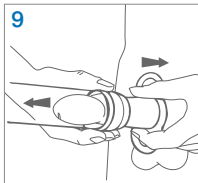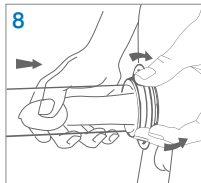NOTE: This lubrication of the O-ring with petroleum jelly is the ONLY use of petroleum jelly.

Do not allow petroleum jelly to come in contact with the constriction rings as it could damage them.

Troubleshooting Guide

Problem	Likely cause	Remedy	Comments
No erection	No vacuum because of air leak caused by poor seal against the penis or seal between tube and pump	Use a smaller constriction ring, more lubricating jelly or trim pubic hair	The problem is often associated with inexperience, it can resolve itself as you become more adept at using the device
Partial erection	Not enough pumping strokes to create a full erection	Use a larger seal sleeve	With continued use the problem often resolves itself
Rapid loss of erection (in less than 5 minutes)	Vacuum effect lost because of air leak, poor seal against the penis, excessive pubic hair or constriction ring too large	Use a smaller constriction ring or more lubricating jelly	Experiment with varying sizes of constriction ring and / or seal sleeves
Delayed loss of erection (after 5 to 10 minutes)	Insufficient tension from constriction rings	Use smaller or a combination of constriction rings	The problem is often associated with inexperience, it can resolve itself as you become more adept at using the device
Pulling the skin of the scrotum	Lubricant on scrotum, improper pumping technique, sleeve too large	Remove lubricant from scrotum, use a smaller sleeve, use the modified pumping technique (ask your physician), or angle cylinder downward during pumping	
Discomfort or pain... during pumping	Pumping too rapidly	Pump slower and continue to use the device	With continued use the problem often resolves itself
from constriction ring	Constriction ring too small	Use a larger constriction ring and continue to use the device	With continued use the problem usually resolves itself
during intercourse	Inadequate lubrication or pressure on a sensitive area	Apply more lubrication or change your intercourse position	
during ejaculation	Long period of abstinence, infection or inflammation of the prostate gland	Continue to use the device	With continued use the problem often resolves itself – if pain or discomfort persists, consult your physician
Redness, irritation or bruising	Pumping too rapidly	Pump slower or use fewer strokes	Consult your physician if this problem persists
Penis feels cold (to you or your partner)	Constriction of blood flow to the penis or lubrication	Wash off lubricant or warm lubricant before use	
Penis pivots on its base	Constriction of blood flow to the penis	If manageable, apply the device after you have a partial erection	

Glossary of Terms

Amputation

Removal of body parts, particularly the arms or legs.

Anticoagulant therapy

Drug treatment that thins the blood to prevent clot formation.

Arteriosclerosis

Hardening of the blood vessels caused by the deposit of a fatty substance in the center of the blood vessels slowing down the blood flow.

Cirrhosis of the liver

Hardening of the liver due to toxic substances or a viral or bacterial infection.

Contraception

Something that prevents pregnancy.

Diabetes

A common condition where you are unable to digest sugars and starches, and which affects the blood glucose levels.

Fibrosis of the penis

The formation of hard areas in the shaft of the penis, which may be caused by repeated injections.

Gangrene

A serious condition that causes death of the body tissue.

Hematoma

Bleeding under the skin causing swelling and very severe bruising.

Hemorrhage

Excessive bleeding.

Impotence due to medication

Side effects of some common drugs used to treat other conditions can cause impotence (i.e. beta blockers, antidepressants).

Libido

The desire to have sexual intercourse.

Manual dexterity

The range of movement in the arm, hand and fingers.

Multiple Sclerosis

A serious neurological condition which also affects the nerve pathways to the penis.

Partial erection

When the erection is not full and firm enough for vaginal penetration.

Pelvic trauma

Injury to the (pelvic area) lower part of the body.

Penile implants

Special rods implanted into the penis to create an artificial erection.

Penile tumescence

Enlargement of the penis due to the inflow of blood.

Peyronies disease

A bend or deviation in the shaft of the penis so that the penis is not straight, caused by a hardening of the penile tunica sheath.

Priapism

A prolonged state of erection from which it is difficult to return the penis to the non erect state.

Prosthesis

An artificial substitute for a missing part.

Retrograde

To flow backwards.

Sickle cell disease

A disease affecting the red blood cells which makes them very fragile and liable to damage. This can cause excessive bruising.

Spinal cord injuries

Injury to the spine often causing paralysis.

Urethral stricture

A narrowing of or blocking of part of the passage between the bladder and the outside of the body.

Vascular disorders

Medical conditions affecting the blood vessels.

Venous leakage

Leaking of blood from the veins.

Warranty

This warranty is offered to you as a purchaser (“**you**”) by the Owen Mumford entity whose details are set out on the relevant device packaging (“**we**” or “**us**”). You cannot transfer this warranty to anyone else. This warranty is in addition to, and does not affect, your rights as a consumer or purchaser under the laws of your country.

This warranty applies for twelve months from the date of purchase (the “**warranty period**”).

If the device is or becomes materially defective during the warranty period, please notify us providing details of the defect and we will provide you with a replacement device. Subject to the laws of your country, we shall have no further liability to you after we have provided you with a replacement device.

When you contact us, we may ask you for details of the batch number or other form of identification of the device in order for us to further investigate the defect. We may also ask you to return the device and any provision of a replacement device is subject to us receiving the defective device from you.

Owen Mumford Inc.

1755 West Oak Commons Court, Marietta, Georgia 30062, USA

T : +1 800-421-6936 **F** : +1 770-977-2866

E : info@owenmumfordinc.com

This warranty will not apply to defects which are attributable to:

- fair wear and tear;
- handling, storage or use of the device other than in accordance with our instructions, or improper handling, storage or use of the device; or
- damage or breakage caused by accident or by external causes beyond our control.

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