



#### **NORTH AMERICA**

Owen Mumford Inc.  
1755 West Oak Commons Court  
Marietta, Georgia 30062, USA  
T : 800-421-6936 F : 770-977-2866  
E : info@owenmumfordinc.com

#### **HEAD OFFICE**

#### **INTERNATIONAL SALES & MARKETING**

Owen Mumford Ltd  
Brook Hill, Woodstock  
Oxfordshire OX20 1TU, United Kingdom  
T : 01993 812021 F : 01993 813466  
E : info@owenmumford.co.uk

#### **UK / IRELAND SALES & MARKETING**

T : 01608 645555 F : 01608 646546  
E : info@owenmumford.co.uk

#### **FRANCE**

T : 02 32 51 88 70 F : 02 32 21 96 95  
E : contact@owenmumford-fr.com

#### **GERMANY**

T : 06026/97750 F : 06026/977529  
E : info@owen-mumford.de

[www.owenmumfordinc.com](http://www.owenmumfordinc.com)

#### **OUR ACCREDITATIONS INCLUDE:**

- ISO13485 Certification
- Medical Device Directive 93/42/EEC
- CMDCAS 13485 (Canadian Health Authority)
- Environmental Management Systems ISO14001



SM2000LT/DIN/0012/01



Drug free treatment  
for erectile dysfunction

# Rapport<sup>TM</sup>

Erectile Dysfunction Management



# Rapport™

## Erectile Dysfunction Management

**Rapport Classic** vacuum therapy device is a user-friendly, effective treatment for erectile problems.

**Rapport Rings** are also available and can be used with or without the **Rapport** pump, for men looking to maintain an erection or avoid premature ejaculation.

### What can cause Erectile Dysfunction?

Erectile dysfunction is a recognized complication of many long term conditions, but there are many factors that can cause erectile dysfunction. Most are directly related to a physical cause, but non-physical conditions such as over-tiredness, stress, tension, depression and some prescribed drugs may also lead to erectile dysfunction.

### What is Vacuum Therapy?

Vacuum Therapy is one of the safest and most effective options for dealing with erectile dysfunction. A pump is used to create a vacuum around the penis which draws blood into the penis, creating a firm erection. A penile ring is slipped over the base of the penis and the pump removed to allow full sexual intercourse to take place. The ring will maintain the erection until removed which must be within 30 minutes. Vacuum therapy does not involve the use of drugs, but is often times recommended as a complimentary therapy to oral and injectable regimens.

**Rapport**, unlike drugs, may be used as often as desired to produce a firm erection in minutes. Because vacuum therapy means a man does not have to take prescribed drugs, there is no risk of interactions with any existing medications.



Scan this code for more information about **Rapport**

“It has changed our love-making and our lives. I feel like a young man again...I only wish now I had sought help years ago.”

Mr. A



# Rapport™ Classic

**Rapport Classic** is a unique vacuum therapy system developed as an effective treatment for erectile problems. This conventional vacuum therapy system creates a vacuum around the penis, quickly producing a satisfactory erection by drawing blood into the penis. The pump is then removed leaving the ring at the base of the penis.

### The Rapport system includes:

- 1 clear plastic vacuum tube
- 1 hand pump with a pressure relief valve
- 5 sizes of penile rings
- 2 sizes of transfer sleeves
- 1 loading cone
- 1 tube of water-soluble lubricating jelly
- 1 demonstration DVD explaining how the device should be used
- Patient instruction booklet



*All items are supplied in a discreet carrying case.*

### Rapport Classic

Product Code: SM 2000

NDC: 08470-2000-01

### Rapport Rings

Product Codes: SM 2203-2209

### Advantages of Vacuum Therapy Devices

Non-drug therapy and therefore no contraindications with other medications

- 95% success rate
- Non-invasive
- An erection can be achieved within minutes and therefore spontaneity for intercourse is not compromised
- Cost effective
- May be used to combat Erectile Dysfunction resulting from any condition
- Can be used in conjunction with other therapies to enhance the erection
- Vacuum therapy benefits the penis by increasing blood flow and can therefore reduce the build up of fibrous tissue