



## THE ABSORBENT COMPOSITE DRESSING

**Alldress®** is an all-in-one dressing that protects the wound, absorbs exudate and creates a moist wound environment.

### Improved Film backing

- Moisture vapor permeable; more breathable
- Maintains a moist environment
- Barrier to moisture and bacteria

### Polyester nonwoven

- Gives stability at application

### Wound contact layer

- Low adherence to wound
- Allows exudate to pass into absorbent pad
- Integrated net increases absorption and retention

### Absorbent pad

- Absorbs exudate

### Water-based adhesive

- Secure yet gentle fixation



## How to use Alldress®



Open the package and remove the dressing.



Remove one of the protection papers to expose the adhesive surface and allow initial fixation of the dressing.



Position Alldress® on the skin and gently remove the second release paper. Firmly smooth the adhesive border to obtain proper adhesion. Do not stretch the dressing when applying!

## How Alldress® works

The porous net is low-adherent and protects against fibers from the absorbent pad entering the wound. The absorbent pad absorbs excess exudate and minimizes the risk of maceration to the surrounding skin, and the gentle adhesive keeps the dressing securely in place. The nonwoven layer provides stability to the dressing structure for ease of application. The semi-permeable film maintains a moist wound environment, prevents wound fluids from passing through the dressing and protects both the wound and the environment from contamination. The smooth surface reduces the friction or potential disturbance of the dressing from bed linens and clothing.

## Benefits of Alldress®


- Low adherence to the wound
- Maintains a moist environment
- Minimizes the risk of maceration
- Moisture proof and bacteria proof film backing
- Protects against contamination
- Smooth film backing prevents friction against clothing and linens



## Areas of use

- Primary dressing for clean wounds and where a barrier to fluids and bacteria is required.
- Secondary dressing for open wounds together with Hypergel®, Normlgel®, Mesalt®, Melgisorb®, and Melgisorb® Ag.


## Wear time: Change PRN

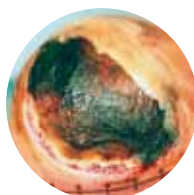

Pressure ulcer



Cavity wound

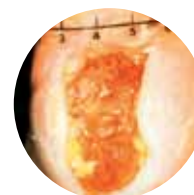

foot ulcer

## Alldress® Ordering Information†

Product Code	Product Size/ Wound Pad Size	Pieces/ box	Pieces/ case	HCPCS
265329	4" x 4" (10 x 10cm)/ 2" x 2" (5 x 5 cm)	10	70	A6203
265349	6" x 6" (15 x 15cm)/ 4" x 4" (10 x 10 cm)	10	100	A6203
265369	6" x 8" (15 x 20cm)/ 4" x 6" (10 x 15 cm)	10	120	A6204

† Packaged sterile in single packs



5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | [www.molnlycke.us](http://www.molnlycke.us) | [www.mhcwoundcare.com](http://www.mhcwoundcare.com)

The Mölnlycke Health Care, Alldress, Hypergel, Normlgel, Mesalt, and Melgisorb, names and respective logos are registered trademarks of Mölnlycke Health Care AB.

Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2011. Mölnlycke Health Care AB. 1.800.882.4582. US007180905

MHC-2011-0364 - 200104 09.11