




CLEANSING SODIUM CHLORIDE DRESSING

Mesalt® effectively debrides moist necrosis (slough) and stimulates the cleansing of heavily draining or infected wounds.

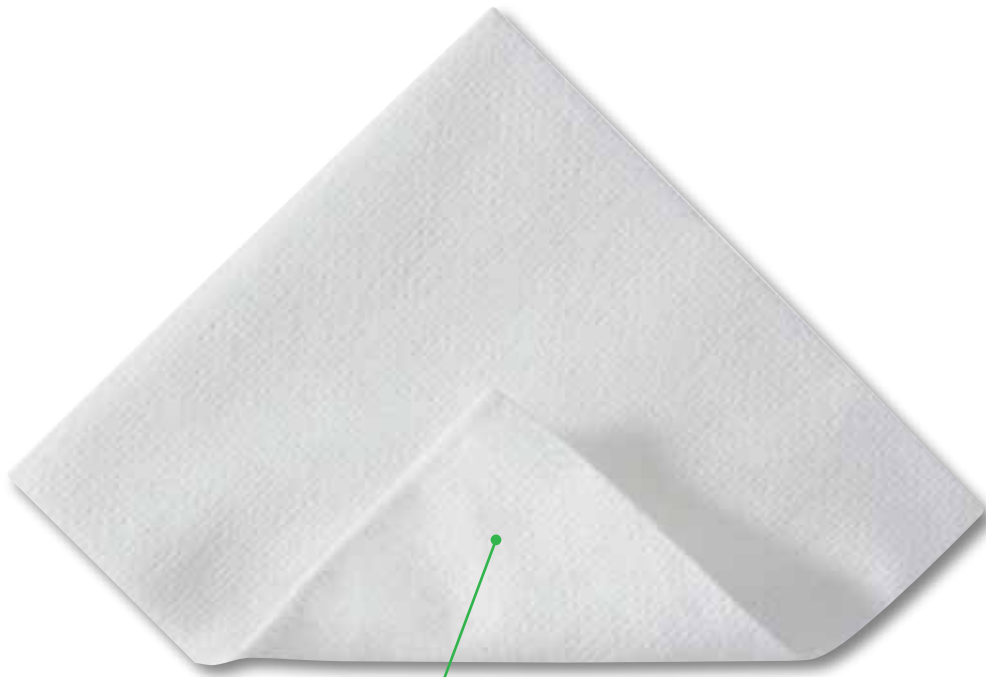


Absorbent viscose/polyester nonwoven •


- Impregnated with sodium chloride




How to use Mesalt®

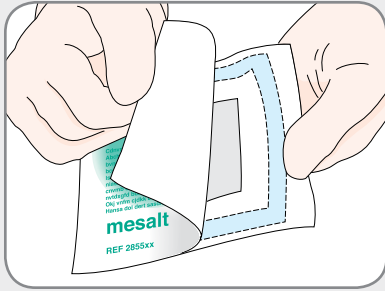

Clean the wound area and debride of any excessive necrotic tissue or eschar. Dry the surrounding skin, but do not dry the surface of the wound. Where clinically indicated, a water-resistant cream or ointment can be used on the surrounding skin.

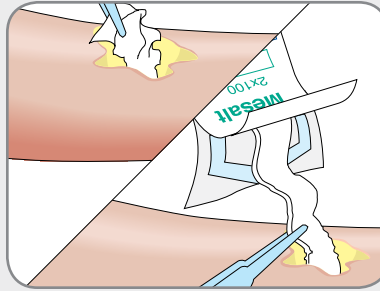

Choose the appropriate **Mesalt®** dressing size, which will:
a) cover a shallow wound or
b) loosely pack a deep wound

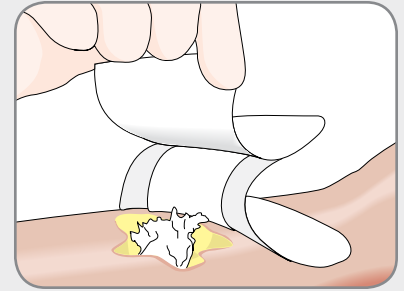

Do not allow the **Mesalt®** to overlap the edges of the wound. Depending on the level of exudate, cover with an appropriate secondary dressing. Where clinically indicated, compression therapy may be used in conjunction with **Mesalt®**.

Mesalt® should be changed when saturated (usually once a day in heavily exuding wounds), or according to good clinical practice. If the condition of the wound deteriorates, discontinue use and consult a healthcare professional.

How Mesalt® works

The exudate from the wound releases the sodium chloride from the dressing. **Mesalt®** stimulates the cleansing of heavily draining or infected wounds in the inflammatory phase by absorbing exudate, bacteria and necrotic material from the wound, thereby facilitating the natural wound healing process.

Benefits of Mesalt®


- Absorbs exudate bacteria and necrotic material (slough)
- Effectively stimulates the cleansing of wounds
- Easy application
- Low potential for skin irritation and allergy

Areas of use

- Heavily draining or infected wounds in the inflammatory phase
- Deep cavity wounds
- Pressure ulcers
- Surgical wounds

Wear time: Daily and PRN

Precautions: **Mesalt®** should not be used on dry wounds with low levels of exudate. **Mesalt®** should not be allowed to come into direct contact with exposed bone or tendon. Infected wounds should be evaluated on a regular basis and treated appropriately.

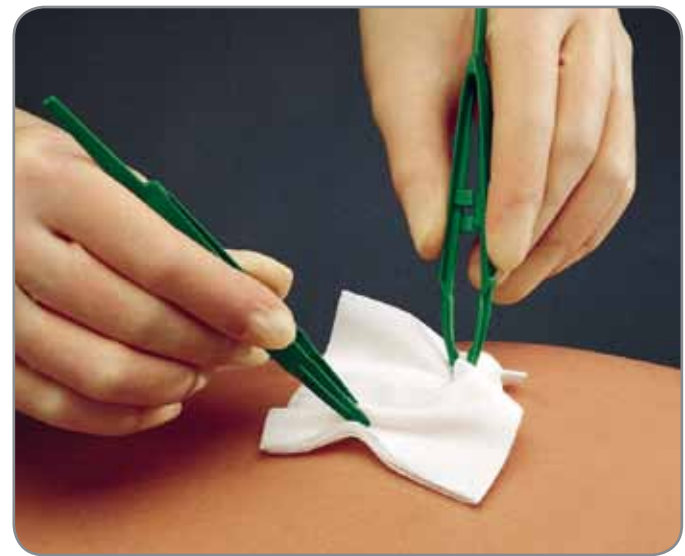

Mesalt® Ordering Information†

Product Code	Size	Pieces/ box	Pieces/ case	HCPCS
285580	4" x 4" [2" x 2" folded]	30	240	A6222
285780	6" x 6" [3" x 3" folded]	30	240	A6223
286099	8" x 8" [4" x 4" folded]	30	180	A6224
285299	3/4" x 39" Ribbon	10	80	A6266

† Packaged sterile in single packs



**MÖLNLYCKE®
HEALTH CARE**

5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | www.molnlycke.us | www.mhcwoundcare.com

The Mölnlycke Health Care, Alldress, Mepore, Mepilex, and Mesalt, names and respective logos are registered trademarks of Mölnlycke Health Care AB. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2011. Mölnlycke Health Care AB. 1.800.882.4582. US006020805

MHC-2011-0126 Rev 1 - 200106 09.11