

Mepilex® Transfer Ag


Safetac®
TECHNOLOGY



Antimicrobial wound contact layer with Safetac® technology

- Effectively manages exudate by transferring fluid into a secondary dressing
- Features **Safetac®** technology for less pain and trauma during dressing changes^{1,2}
- Provides a rapid and sustained antimicrobial effect³⁻⁷
- Uniquely conformable, thin foam construction

- The absence of a backing film allows exudate to transfer into a secondary dressing, that can be changed as needed, while **Mepilex® Transfer Ag** remains in place and effective for up to 14 days

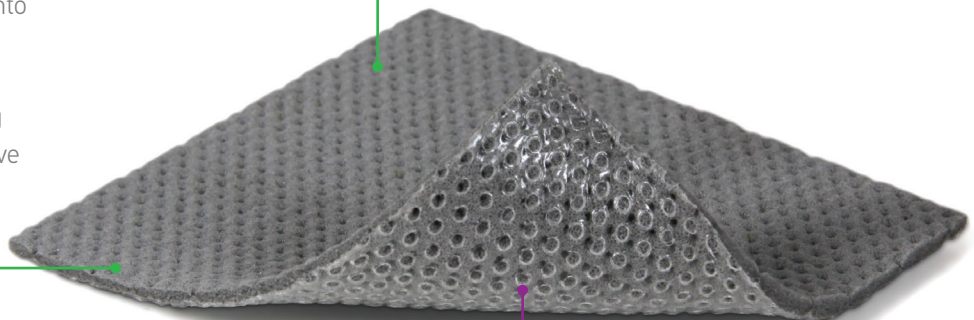


- The antimicrobial silver sulphate impregnated, polyurethane foam transfers exudate into a secondary dressing

- Exclusive **Safetac®** technology supports less painful and less traumatic dressing changes^{1,2}

Safetac® technology. Less pain and less trauma.^{1,2}

Dressings with **Safetac®** technology are clinically proven to hurt less.¹ That means less stress⁹ and fewer disturbances to the wound, which can mean faster healing⁹ and resulting lower costs.¹⁰ **Safetac®** technology is available exclusively on Mölnlycke Health Care dressings.

For more information visit www.molnlycke.us

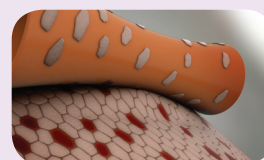


Skin stripping may occur with traditional adhesive



Minimal skin stripping with Safetac® technology

How to apply Mepilex® Transfer Ag

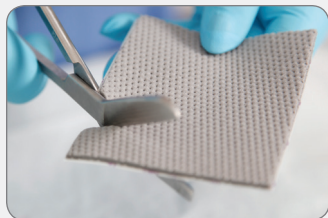

1. **Preparation** - Cleanse the wound and dry the surrounding skin thoroughly. If needed, cut Mepilex® Transfer Ag to the appropriate size or shape.



2. **Apply Mepilex® Transfer Ag** - Remove the release film, and apply the adherent side to the wound allowing at least 2-5 cm overlap on dry skin. Do not stretch the dressing during application.



3. **Apply secondary dressing** - Apply an absorbent secondary dressing that is appropriate for the exudate level. The secondary dressing should extend 2-5 cm onto the periwound skin.



4. **Apply fixation** - Use an appropriate fixation to hold the dressings in place, e.g. Tubifast®, Dermafit™, etc.

How Mepilex® Transfer Ag works

Mepilex® Transfer Ag is a thin and conformable, antimicrobial wound contact layer. The Safetac® technology layer ensures direct contact to the wound and the surrounding skin, including uneven surfaces. As the Safetac® technology layer seals around the wound, the foam structure of Mepilex® Transfer Ag allows exudate to move vertically into a secondary absorbent dressing. Mepilex® Transfer Ag maintains a moist wound environment in combination with an appropriate secondary dressing.

Mepilex® Transfer Ag inactivates wound related pathogens within 30 minutes^{3,4} with a sustained effect up to 14 days.^{5,6,7} By reducing the number of microorganisms, Mepilex® Transfer Ag may also reduce odor.

Areas of use

Mepilex® Transfer Ag is indicated for low to high exuding wounds including difficult-to-dress areas where an antimicrobial effect is needed, including:

- traumatic wounds
- partial thickness burns
- surgical wounds
- foot and leg ulcers
- pressure ulcers

Mepilex® Transfer Ag can be used under compression bandaging.

Wear time: Up to 14 days

Benefits of Mepilex® Transfer Ag

- Transfers exudate away from the wound
- Minimizes pain and trauma at dressing change^{1,2}
- Effective, sustained release of silver provides rapid antimicrobial effect within 30 minutes^{3,4} that is sustained for up to 14 days^{5,6,7}
- Promotes patient comfort during wear
- Conforms well to body contours
- May be cut to desired shapes
- Effective under compression bandages

Mepilex® Transfer Ag Ordering Information†

Product Code	Size	Pcs/box	Pcs/case	HCPCS
394190	4" x 5" (10 x 12.5 cm)	5	50	A6210
394890	6" x 8" (15 x 20 cm)	10	50	A6210
394590	8" x 20" (20 x 50 cm)	2	12	A6211
394990	4" x 47" (10x120 cm)	1	9	A6211

† Packaged sterile in single packs

Precautions

Please refer to the package insert for use for details.

References

1. White R. A multinational survey of the assessment of pain when removing dressings. Wounds UK, 2008.
2. Dykes PJ et al. Effects of adhesive dressings on the stratum corneum of the skin. J Wound Care 2001; 10(2):7-10.
- 3-7. Data on file.
8. Upton D. et al. The Impact of Atraumatic Vs Conventional Dressings on Pain and Stress in Patients with Chronic Wounds. Submitted and approved for publication. Journal of Wound Care, 2012.
9. Upton D. et al. Pain and stress as contributors to delayed wound healing. Wound Practice and Research, 2010.
10. Butcher M. et al. Quantifying the financial impact of pain at dressing change. Psychological impact of pain in patients with wounds. Wounds UK, 2011.



We're here to help you, when you need us. Call your Mölnlycke® Health Care Representative or Regional Clinical Specialist.

1-800-843-8497 | www.molnlycke.us | 5550 Peachtree Parkway, Suite 500, Norcross, GA 30092