



Secure and gentle fixation tape with Safetac® technology

Mepitac® tape is atraumatic to the skin upon removal and is ideal fixation for patients with fragile skin or at risk for those reactions to traditional tapes and adhesives.

Safetac® technology layer


- Reduces pain and trauma upon removal
- Leaves no residue upon removal

Polyurethane backing film

- Breathable
- Moisture proof

Knitted fabric

- Conformable
- Easy to cut

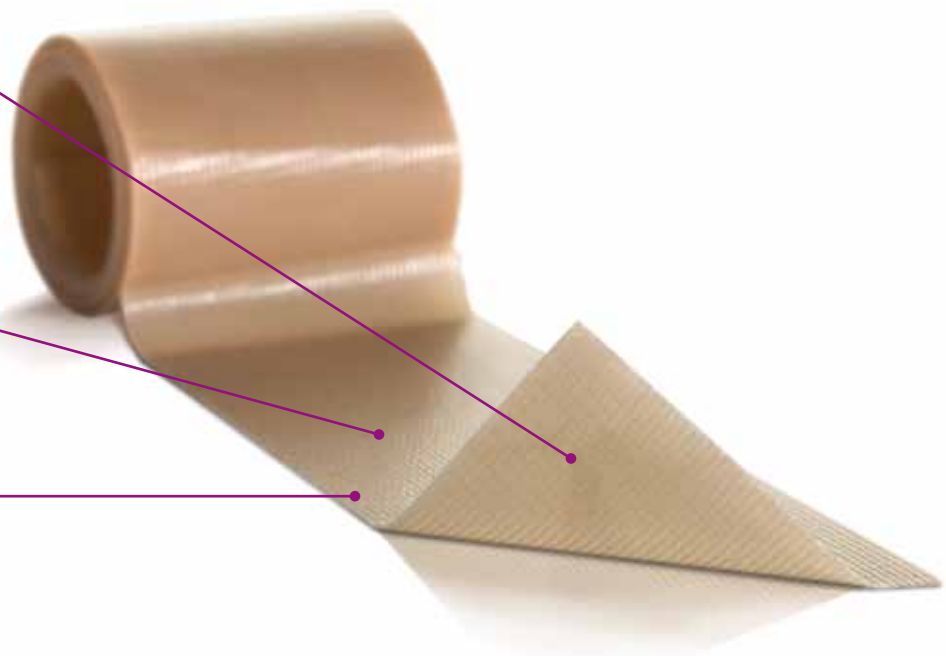

Safetac® technology. Less pain and less trauma.

Safetac® is a patented adhesive technology that minimizes pain to patients and trauma to wounds. Safetac® technology is available exclusively on Mölnlycke Health Care dressings, including Mepilex® wound care dressings, Mepitel®, Mepiform® and Mepitac®.

For more information visit www.molnlycke.us




Painful skin stripping with traditional adhesive




No skin stripping with Safetac® technology


How to use Mepitac®

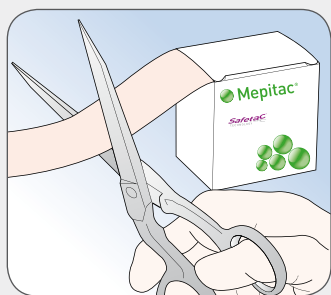

Make sure the skin is clean, dry and free from any ointment, etc. Cut off the desired length of Mepitac® using scissors.

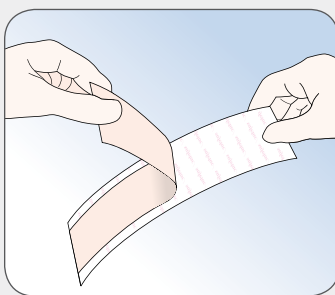

Remove the protective release film.

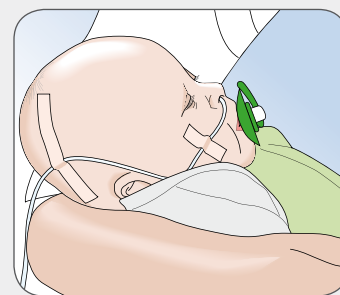

Apply Mepitac® without stretching. If inspection or repositioning is necessary, Mepitac® can be lifted and reapplied.

How Mepitac® works

Mepitac® is highly conformable and easy to apply and remove. The Safetac® layer ensures secure fixation and atraumatic removal from skin and hair. If inspection or repositioning is necessary, Mepitac® can be lifted and re-applied. It is an ideal fixation solution for patients with fragile and sensitive skin or with allergies to traditional adhesives. Mepitac® is suitable for patients requiring repeated application and removal of tape over the same area (e.g., neonates and dialysis patients).

Areas of use

- Fixation of medical devices such as drains, tubes, probes, electrodes, IV cannulae, and dressings
- Skin protection when used under devices (e.g., tubes).
- May be repositioned

Note: The adhesion of Mepitac® may decrease when used in humid conditions such as incubators or on patients who are diaphoretic. Mepitac® is not suitable for fixation of arterial catheters and arterial cannulae.



Benefits of Mepitac®

- Atraumatic to the skin on removal
- Conforms well to body contours
- Secure fixation
- Leaves no residue
- Breathable, moisture proof film backing
- May be repositioned
- Suitable for patients with fragile skin, sensitive skin or allergies to traditional adhesives



Mepitac® Ordering Information†

Product Code	Size	Rolls/box	HCPCS
298300	3/4" x 118" (2 x 300 cm)	12	A4452
298400	1.5" x 59" (4 x 150 cm)	12	A4452

† Packaged sterile in single packs

Safetac®
TECHNOLOGY

MÖLNLYCKE®
HEALTH CARE

5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | www.molnlycke.us | www.mhcwoundcare.com