



LIMB TUBULAR BANDAGES

- Tubipad tubular bandages provide non-invasive, mechanical support
- Convenient and practical
- Minimizes friction and pressure on the skin

Washable and
reusable

Helps to minimize
friction and
pressure on the
skin



Easy to apply
and remove

Remains
securely in
place

How Tubipad works

Tubipad is manufactured from a layer of polyurethane foam bonded to a length of Tubigrip™ tubular elasticated bandage. The soft inner layer minimizes pressure and friction on the skin, while the outer Tubigrip layer holds the Tubipad securely and comfortably in position.

The tubular construction ensures that Tubipad is easy to apply and remove.

Benefits of Tubipad

- Minimizes friction and pressure on the skin
- Remains securely in place without tapes or pins
- Easy to apply and remove
- Available in a range of sizes for specific applications
- Convenient and practical
- Washable and reusable

Washing instructions



Warning: Contains natural rubber latex.

Tubipad



Indications for use

Tubipad is intended to help prevent pressure and friction sores on heels, elbows and knees.

Material properties

- Polyurethane foam
- Elasticated support bandage consisting of:
 - Cotton
 - Elastodiene (contains natural rubber latex)
 - Polyamide

Directions for use

1. Choose appropriate size of Tubipad.
2. Pull into position over the area to be protected, with foam padding next to the skin.

Tubipad Assortment

Art. No	Size	Application
4580	Medium	Adult heels, elbows, knees
4581	Large	Broad or swollen heels or knees

Tubipad Limb Bandages are supplied in individually packed 4m rolls

Mölnlycke Health Care
1-800-494-5134
www.molnlycke.ca

The Mölnlycke Health Care name and logo and Tubipad™ are registered trademarks of Mölnlycke Health Care AB.
Gamlestadsvägen 3C, Göteborg, Sweden 40252.



CA013380904EN