



Knowing now matters.™



Alere™ Influenza A & B Test

The Alere Influenza A & B Test provides a simple, rapid method as an aid in the differential diagnosis of influenza A & B using nasal swab specimens. This easy-to-use format and rapid results allow for point-of-care testing where it can provide information to assist with treatment and patient management decisions.

Features of the Alere Influenza A & B Test

- Results in 10 minutes
- Nasal swab sample
- CLIA-waived
- Dual reimbursement¹

Benefits of Rapid Diagnosis

- Guide appropriate therapy
- Avoid unnecessary antibiotic use
- Empower timely treatment decisions




To learn more about rapid detection of **influenza A & B**, contact your local Account Executive

Alere™ Influenza A & B

Test Performance*

Format: Dipstick

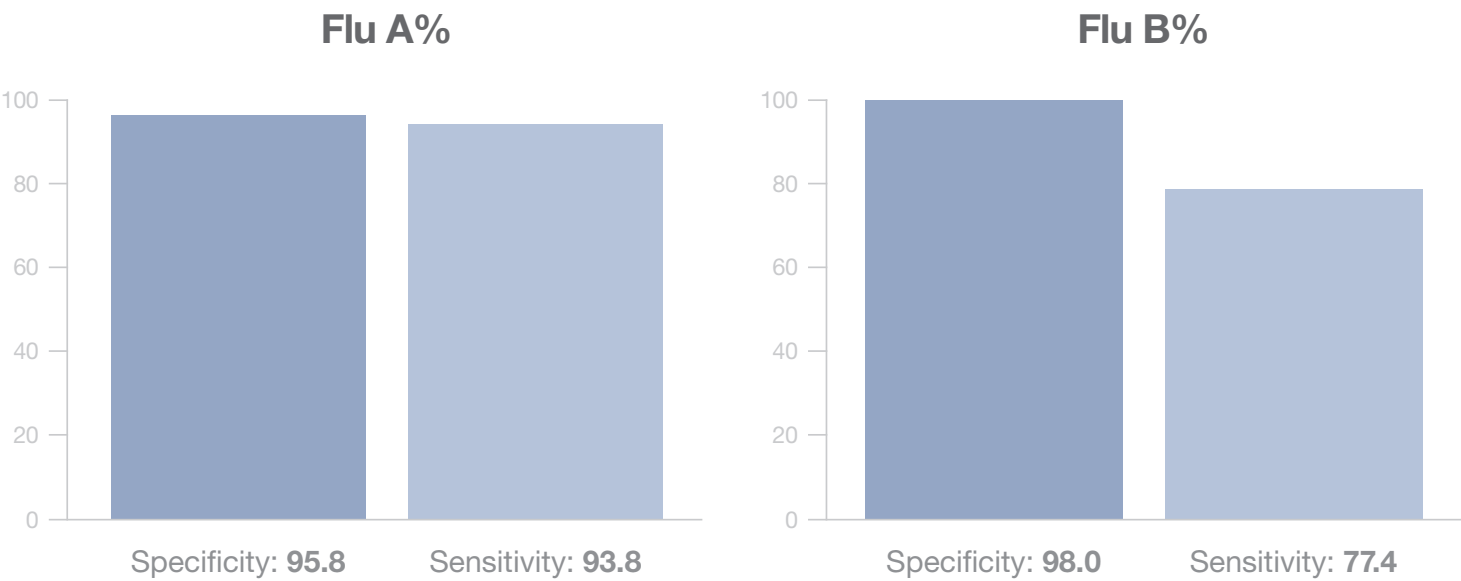

Easy to Use

1

Empty all of the liquid in an R1 vial into coated Reaction Tube.

2

Place the swab in the Tube and mix it in the liquid for about 10 seconds.

3

While squeezing the Tube around the swab head, remove the swab from the Tube.

Add a Test Strip.


4

Wait 10 minutes, then read the result.

10 min

For complete instructions for use, see the package insert.

Easy to Read

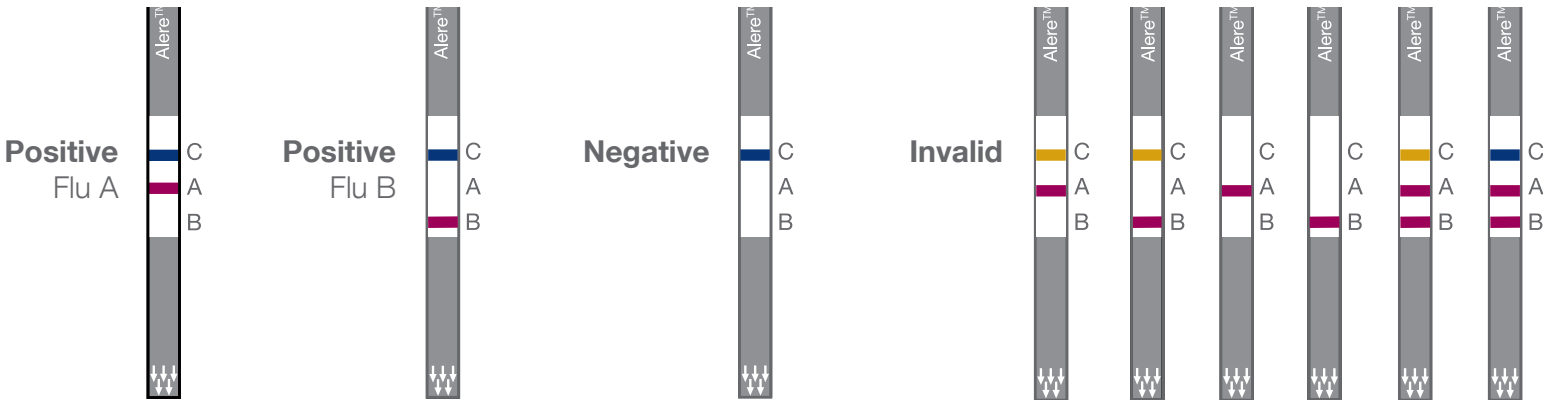

1. Actual reimbursement may vary.
* Versus viral culture.

© 2016 Alere. All rights reserved. The Alere Logo, Alere and Knowing now matters are trademarks of the Alere group of companies. CPT is a registered trademark of the American Medical Association. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. 12000333E-02 11/16



Call 1.877.441.7440 or visit alere.com today

Order Information

Alere Influenza A & B Test
CLIA-waived
25 Test Kit
Order No. 412-000
CPT® Code: 87804/QW