


SILENT KNIGHT® Pill Crusher In-Service

The Silent Knight is a simple, safe and efficient way to crush pills from solid to fine powder. When following the below steps and procedure it is possible to crush a single pill or multiple pills quickly and effectively.

IN-SERVICE STEPS:

1. Place pill(s) into Silent Knight Pill Crusher Pouch.
Lift handle and place pouch down between the crusher plates to initiate crushing.
2. Press handle down to crush pill(s). Lift handle and pull pouch upward to the middle of the crusher plates.
3. Then press the handle down to activate grinding movement.
Press handle down several times to grind pill(s) into fine powder. Repeat if needed.
4. Remove the pouch and add the medication to an appropriate serving medium for administration.

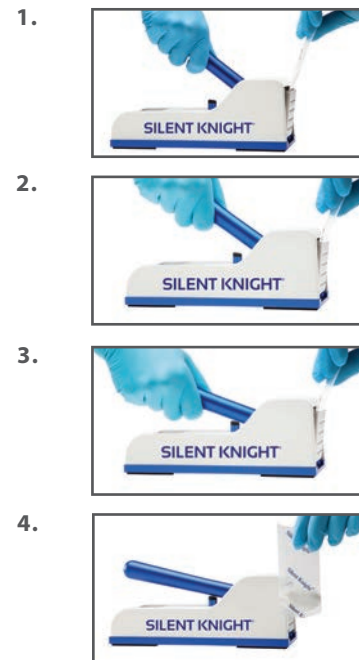

SILENT KNIGHT EXCLUSIVE PILL CRUSHER POUCH

The Silent Knight does not function alone. Its companion item, the patented Silent Knight Pill Crusher Pouch, is essential to guarantee maximum performance while reducing cost, waste, malfunction and lost time.

The pouch's distinctive curved bottom structure assures full recovery of crushed medication. The thick, durable construction effectively reduces pharmaceutical cross contamination and eliminates loss of medication.

CLEANING & MAINTENANCE

The Silent Knight is made entirely of non-rusting materials and may be cleaned regularly with a damp cloth. A facility approved disinfectant wipe may also be used when indicated.