

# SILENT KNIGHT® Pill Crusher

## Cleaning and Maintenance Instructions

The Silent Knight Pill Crusher is made entirely of non-rusting materials and may be cleaned regularly with a damp cloth. A facility approved disinfectant wipe may also be used when indicated.

In instances when the Silent Knight Pill Crusher requires a more thorough cleaning, please follow the instructions below:

### Step #1

1. Turn over pill crusher so the blue base faces up. With a Phillips head screwdriver, remove each of the five (5) screws. (Fig. 1)

Fig. 1



### Step #2

1. Once the screws are removed, keep all five (5) screws together. (Fig. 2) *NOTE: The center screw in the row of three above the label is different. This is the only screw that should be used in this location.*
2. Turn the pill crusher back to upright position being careful to hold both the blue base and the top cover in place.
3. Once upright, lift up on the blue handle and lift off the top cover. It should lift off over the blue handle. (Fig. 3)

Fig. 2



Fig. 3



### Step #3

1. Using a damp cloth, wipe clean the Silent Knight Pill Crusher using a normal detergent and water. Wipe down with dry cloth. (Fig. 4)

Fig. 4



### Step #4

1. Place top tray back on to the base by guiding over the blue handle. (Fig. 5)
2. Turn the pill crusher so the blue base faces up being careful to hold both the blue base and the top cover in place.

Fig. 5



### Step #5

1. With a Phillips head screwdriver, screw in each screw (clockwise) until screw is tight. (Fig. 6)

Fig. 6



When possible, please have a facility engineer or other authorized personnel remove the top of the pill crusher for cleaning.

Any further disassembly of the pill crusher will result in voiding the lifetime warranty.