


# OBP<sup>®</sup> MEDICAL Fluid Management PACK<sup>™</sup>

The Premier Fluid Management Kit For Hysteroscopy

Remove Fluid Management PACK contents from tyvek bag and place on sterile surface. Unfold under buttocks drape and remove covering from sticky tape on the top underside of the drape. Stick under buttocks drape firmly to patient table allowing drape to hang down with collection bag facing outward (see example "A" below).

Example A

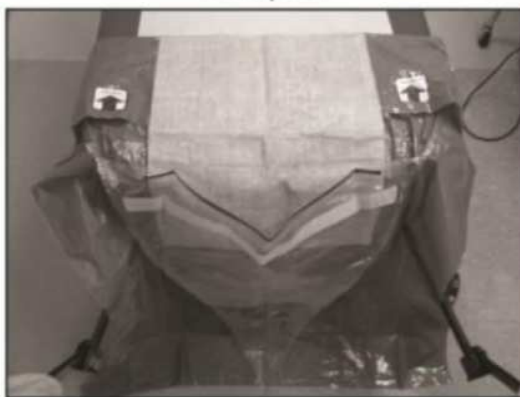

Attach OutflowBAG tubing to the outflow port of hysteroscope. Place collection bag on floor or in position that allows fluid to flow downward (see example "B" below).

Example B



Attach cysto tubing to inflow port of hysteroscope using luer-lock adaptor. Spike fluid bag with opposite end of tubing to allow fluid to flow.