


Hygie® Extra-absorbent pad



<u>Tender description</u>	Hygie extra-absorbent pad Absorption capacity : 1000 ml Dimensions : 9.5" x 8" – 24 x 20 cm No expiry date 100% made from recycled materials Made in Canada 
Product code	Case code: SA-PADS-8X95-000
Présentation	Single use disposable pad which gelifies bodily fluids in less than 30 seconds. It limits the spread of pathogens thereby reducing risks of nosocomial infections. It makes the collection and the transportation of bodily fluids safer. Less risks of splashing and spilling.
Colour	White.
Dimensions	Length = 9.5"- (24 cm). Width = 8"- (20 cm).
Composition	Extra absorbent pad 100% recycled composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. No latex and no phthalates.
Shelf life	No expiry date.
Efficiency	Absorption of 1000 ml of liquids in less than 30 seconds.
Manufacturing process	Compression and heating of fabric and cellulose pulp mixed with a super absorbent substrate.
Use	It is used in bedpan, vomit and urinal Hygienic Cover, as well as in any support intended to collect fluids. Particularly recommended for isolation case.
Instructions for use	Open the packet and take out a pad. Place it at the bottom of the support intended for the collection of fluids. Let the patient relieve himself/herself into the support. Wait for the gelification of the liquids (± 1000 ml within seconds). Discard in the appropriate location. Refer to the usage instructions for more detail.
Precautions for use	Single-use product. Do not reuse. Keep away from children.
Disposal	Dispose of product in regular waste or according to the protocol of the institution. Don't dispose in toilet.
Benefits	Increases hygiene, disposable product, saves time, reduces odours, simple and risk-free disposal. Prevention of nosocomial infections.
Identification	Each product is identified with a lot number.
Regulatory	FDA: class I medical device exempt from 501(k). CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.
Risks/Safety	Very slight risk of ocular or respiratory irritation in case of extended contact with the extra absorbent pad. Safety data sheet is available upon request.
Handling/Storage	Store in a clean and dry place, away from sun.

Packaging	Unit	Bag of 40 extra absorbent pads	Pad: 24 x 20 cm 9.5 x 8 in
	Case	10 bags of 40 pads per case or 400 pads per case	60 x 42 x 27 cm / 17 kg 23.5 x 16.5 x 10.75 po / 37.5 lb
	Pallet	20 cases per pallet or 8000 extra absorbent pads	126 x 102 x 121 cm / 362 kg 49.5 x 40 x 47.75 pouces / 800 lb