




## Hygie<sup>®</sup> Super-absorbent pad

### Tender description

**Hygie super-absorbent pad**  
**Absorption capacity : 600 ml**  
**Dimensions : 9.5" x 5" – 24 x 13 cm**  
**No expiry date**  
**100% made from recycled materials**  
**Made in Canada** 

### Product code

Case code: SA-TAMP-ABSO-400

### Présentation

Single use disposable pad which gelifies bodily fluids in less than 30 seconds. It limits the spread of pathogens thereby reducing risks of nosocomial infections. It makes the collection and the transportation of bodily fluids safer. Less risks of splashing and spilling.

### Colour

White.

### Dimensions

Length = 9.5"- (24 cm). Width = 5"- (13 cm).

### Composition

Super absorbent pad 100% recycled composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. No latex and no phthalates.

### Shelf life

No expiry date.

### Efficiency

Absorption of 600 ml of liquids in less than 30 seconds.

### Manufacturing process

Compression and heating of fabric and cellulose pulp mixed with a super absorbent substrate.

### Use

It is used in bedpan, vomit and urinal Hygienic Cover, as well as in any support intended to collect fluids. Particularly recommended for isolation case.

### Instructions for use

Open the packet and take out a pad.  
 Place it at the bottom of the support intended for the collection of fluids.  
 Let the patient relieve himself/herself into the support.  
 Wait for the gelification of the liquids (± 600 ml within seconds).  
 Discard in the appropriate location.  
**Refer to the usage instructions for more detail.**

### Precautions for use

Single-use product. Do not reuse. Keep away from children.

### Disposal

Dispose of product in regular waste or according to the protocol of the institution. Don't dispose in toilet.

### Benefits

Increases hygiene, disposable product, saves time, reduces odours, simple and risk-free disposal.  
 Prevention of nosocomial infections.

### Identification

Each product is identified with a lot number.

### Regulatory

**FDA:** class I medical device exempt from 501(k).  
**CE marking:** Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

### Risks/Safety

Very slight risk of ocular or respiratory irritation in case of extended contact with the super absorbent pad. Safety data sheet is available upon request.

### Handling/Storage

Store in a clean and dry place, away from sun.

### Packaging

Unit	Bag of 20 super absorbent pads	Pad: 13 x 24 cm 5 x 9.5 in
Case	20 bags of 20 pads per case or 400 pads per case	26.7 x 21.6 x 59.7 cm / 8.1 kg 10.5 x 8.5 x 23.5 in / 18 lbs
Pallet	40 cases per pallet or 16000 super absorbent pads	122 x 107 x 120 cm / 349 kg 48 x 42 x 47.25 in / 770 lbs