



GO VOM[®] – Graduated Vomit bag

Tender description

Hygienic Cover – graduated vomit bag with a super-absorbent pad

Absorption capacity : 300 ml

Cover dimensions : 15.5" x 6" – 39.5 x 15 cm

No expiry date

Eco-friendly product

Made in Canada 

Product code

Case code: SA-HYGI-GOVO-000

Presentation

Graduated vomit bag with a cardboard cone containing a super absorbent pad which gelifies bodily fluids and reduces odours. It allows vomit collection.

Cover dimensions

Length (cone included): 15.5" – (39.5 cm) x Width: 6" - (15 cm). Thickness : 38µm.

Composition

Transparent polyethylene bag with a cardboard cone, graduated in ml. Super absorbent pad 100% recycled composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. Packaging made of recycled materials. No latex and no phthalates.

Shelf life

No expiry date.

Efficiency

Absorption of 300 ml of liquid in less than 30 seconds.

Manufacturing process

Heat welding of plastic film.

Use

For any person, men or women, experiencing vomiting, whether in a health care institution, at home or during a trip. Particularly recommended for oncology services, dialysis, post-operative recovery room, as well as for emergency services, such as ambulances.

Instructions for use

Open the box and take out a bag.

Open the cone.

Vomit into the bag.

Wait for the gelification of the fluids (± 300 ml within seconds).

Discard in the appropriate location.

Refer to the usage instructions for more detail.

Precautions for use

Single-use product. Do not reuse. Keep away from children.

Disposal

Dispose of product in household waste. Don't dispose in toilet.

Benefits

Increases hygiene, disposable product, saves time, reduces odours, simple and risk-free disposal.

Improved replacement of the kidney tray or paper bag.

Identification

Each product is identified with a lot number.

Quality

Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501: *Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.*

Regulatory

FDA: class I medical device exempt from 501(k).

CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

Risks/Safety

Very slight risk of ocular or respiratory irritation in case of extended contact with the super absorbent pad. Safety data sheet is available upon request.

Handling/Storage

Store in a clean and dry place, away from sun.

Packaging

Unit	Box of 35 GOVOM bags	20.3 x 11.4 x 18.4 cm / 1 kg 8 x 4.5 x 7.25 in / 2.2 lbs
Case	12 boxes of 35 GOVOM bags per case. Total of 420 GOVOM bags.	60 x 27 x 42 cm / 13 kg 23.5 x 10.75 x 16.5 in / 28.4 lbs
Pallet	20 cases per pallet or 240 boxes of 35 GOVOM bags	126 x 102 x 121 cm / 280 kg 49.5 x 40 x 47.75 in / 618 lbs