


Cylindrical Hanging Scale KCY01

Urine output measurement method with the use of hygienic covers

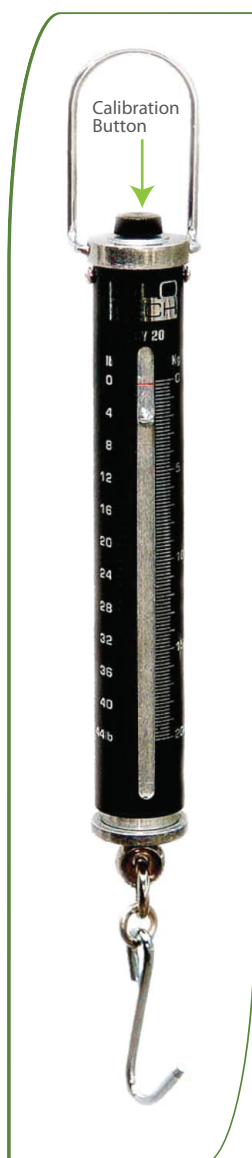

Procedure for measuring urine output in a hygienic cover for regular or orthopedic bedpans

1. Close the hygienic cover according to the established procedure.
2. Hang the bag on the hook.
3. Hold the scale so that the mobile stem is free.
4. Read the measurement.
5. Dispose of the hygienic cover in the appropriate location.
6. Disinfect the scale with a disinfecting wipe.
7. Wash your hands.
8. Put the scale away in designated area.

Calibration

1. Make sure the scale is correctly calibrated or recalibrate as needed.
2. Adjust the calibration as follows :
 - Hang the scale
 - Place the empty hygienic cover (with pad) on the hook
 - Adjust using the calibration button so as the align the weight indicator to the "0" mark.
This will enable the scale to deduct the weight value of the hygienic cover and pad when taking the output measurement.



Features

- Max. Capacity: 1 kg / 2.7 lb
- Graduation: 10 g / 0.5 oz
- 30mm (1.175 inches) tall
- Easy to read single sided dual kg and lb dial
- Main Scale 100g
- Secondary Scale 10g

To order:

Product code: BA-KCY0-1000-000

www.hygie.com

Toll Free : 1 (866) 588-2221

Fax : (450) 444 6222

Canada, USA