



Hygienic Cover[®] for pail / orthopedic bedpan

Tender description

Hygienic Cover for pail and orthopedic bedpan with a super-absorbent pad

Absorption capacity : 600 ml

Cover dimensions : 20.5" x 15.75" – 52 x 40 cm

No expiry date

Eco-friendly product

Made in Canada 

Product code

Case code: SA-HYGI-CLSB-000

Presentation

Hygienic cover[®] – pail bag containing a super-absorbent pad which gelifies bodily fluids and blocks odours. It is used in association with the pail/orthopedic bedpan.

Cover dimensions

Length = 20.5" - (52 cm). Width = 15.75" - (40 cm). Thickness = 38µm.

Composition

Polyethylene bag equipped with ties. Super absorbent pad 100% recycled composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. Packaging made of recycled materials. No latex and no phthalates.

Shelf life

No expiry date.

Efficiency

Absorption of 600 ml of liquids in less than 30 seconds.

Manufacturing process

Heat welding with insertion of sliding ties.

Use

For any person, men or women, who needs a pail or an orthopedic bedpan whether in a health care institution or at home. Particularly recommended for isolation case.

Instructions for use

1. Open the packet and take out a roll of bags.
2. Unroll the first bag and tear it along the cutting line.
3. Open the bag and verify the presence of the super-absorbent pad at the bottom of the bag.
4. Place the bag on the orthopedic bedpan.
5. Let the patient relieve himself/herself into the bag.
6. Wait for the gelification of the fluids (± 600 ml within seconds).
7. Close the bag, tie it and discard in the appropriate location.

Refer to the usage instructions for more detail.

Precautions for use

Single-use product. Do not reuse. Keep away from children.

Disposal

Dispose of product in regular waste or according to the protocol of the institution. Don't dispose in toilet.

Benefits

Offers protection for orthopedic bedpans. Increases hygiene, disposable product, saves time, reduces odours, simple and risk-free disposal. Limits the spread of pathogens.

Identification

Each product is identified with a lot number.

Quality

Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501: *Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.*

Regulatory

FDA: class I medical device exempt from 501(k).

CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

Risks/Safety

Very slight risk of ocular or respiratory irritation in case of extended contact with the super absorbent pad. Safety data sheet is available upon request.

Handling/Storage

Store in a clean and dry place, away from sun.

Packaging

Unit	Box of 20 orthopedic bedpan bags	20 x 11.4 x 11.4 cm / 0.81 kg 8 x 4.5 x 4.5 in / 1.78 lbs
Case	20 boxes of 20 bags (400)	60 x 27 x 42 cm / 20.6 kg 23.5 x 10.75 x 23.5 inches / 45.5 lbs
Pallet	20 cases per pallet or 400 boxes of 20 orthopedic bedpan bags	126 x 102 x 121 cm / 435 kg 49.5 x 40 x 47.75 inches / 960 lbs