



Dispenser for Hygienic Covers®

Tender description

Dispenser for Hygienic Covers – bedpan, urinal et vomit bags
Bedpan washer compatible
Recyclable
Made in Canada 

Product code

Case code: DI-SACS-0000-000

Presentation

Dispenser for Hygienic Covers in polypropylene. It is used in association with Hygienic Cover for bedpan, commode chair, urinal support and vomit support. Can be hanged on a wall or assembled on our accessories plate for Hygie commode chairs (Hygie Classic or Hygie Alu-Classic).

Colour

Transparent.

Support dimensions

Length = 9" - (23 cm) x Width = 7.5" - (19 cm) x Height = 9" - (23 cm).

Composition

Polypropylene. No latex and no phthalates.

Shelf life

No expiry date.

Efficiency

Can hold up to 20 Hygienic Covers.

Manufacturing process

Injection moulding.

Use

For any wards using Hygienic Covers.

Instructions for use

Open the lid.
 Insert Hygienic Covers inside the dispenser.
 Bring out a part of the first bag by the central opening of the lid.
 Close the lid.
Refer to usage instructions for wall mount

Precautions for use

None.

Cleaning / Sterilization

The dispenser must be regularly cleaned and when soiled. It is recommended to use a disinfectant detergent or an enzyme product associated with a mechanical rubbing action. Follow the instructions on disinfectants to maximize their efficiency (dilution, contact time ...). It can also be cleaned in a bedpan washer without exceeding the temperature of 203°F or 95°C.

Disposal

The dispenser is recyclable (Plastic #5 - PP).

Benefits

Opening on the top, easy Hygienic Covers dispensing.
 Durable lid with closing mechanism.
 Sold with a wall bracket for easy wall mount and removal.

Identification

Each case of dispenser is identified with a lot number.

Regulatory

FDA: class I medical device exempt from 501(k).

Risks / Safety

This product was evaluated and is not considered hazardous based on the hazard criteria established by the OSHA (Occupational Safety and Health Administration). Safety data sheet is available upon request.

Handling/Storage

Store in a clean and dry place, away from sun.

Packaging

Unit	Not packaged individually	23 x 19 x 23 cm 9 x 7.5 x 9 po
Case	6 dispensers with bracket per case	40.5 x 60 x 32 cm / 2.5 kg 16 x 12.5 x 23,75 po / 5.5 lb
Pallet	18 cases per pallet or 108 dispensers	122 x 102 x 133 cm / 67.5 kg 48 x 40 x 52¼ po / 149 lb