



## Hygienic™ garbage bags

### Tender description

**Garbage bags**

**Cover dimensions** : 24.5" x 18" – 62.2 x 45.7 cm

**No expiry date**

**Eco-friendly product**

**Made in Canada** 

### **Product code**

Case code: SA-HYGI-BAG1-400

### **Presentation**

White garbage bags with drawstring, ultra-resistant to any type of waste. Designed to be combined with the Hygie super or extra absorbent pad to make your own hygienic cover. After assembly it can be used with the Hy21® bedpan support or with a standard bedpan or commode chair to collect bodily fluids such as urines and feces. Pads are sold separately.\*

### **Cover dimensions**

Length = 24.5"- (62.2cm). Width = 18"- (45.7cm). Thickness = 38µm.

### **Composition**

Polyethylene bag equipped with ties. No latex and no phthalates. Packaging made of recycled materials.

### **Shelf life**

No expiry date.

### **Efficiency**

Bag capacity = 14 liters. Absorption capacity = 600 ml with a super absorbent pad (size 9.5 x 5 in) / 1000 ml with an extra absorbent pad (size 9.5 x 8 in).

### **Manufacturing process**

Heat welding with insertion of sliding ties.

### **Use**

For any person, men or women, who needs a bedpan and/or commode chair whether it's in an institution or at home. Particularly recommended for isolation case.

### **Instructions for use**

1. Open the packet and take out a roll of bags.
2. Unroll the first bag and tear it along the cutting line.
3. Open the bag and place a super/extra absorbent pad at the bottom of the bag.
4. Place the bag on the Hy21® bedpan support, the bedpan or the commode chair.
5. Let the patient relieve himself/herself into the bag.
6. Wait for the gelification of the liquids.
7. Close the bag, tie it and discard.

**Refer to the usage instructions for more detail.**

### **Precautions for use**

Single-use product. Do not reuse. Keep away from children.

### **Disposal**

Dispose of product in regular waste or according to the protocol of the institution. Don't dispose in toilet.

### **Benefits**

When combined with an absorbent pad :

- Protection for Hy21® bedpan supports, standard bedpans or commode chairs.
- Increases hygiene, saves time, reduces odours, simple and risk-free disposal.
- Limits the spread of pathogens. Single-use product.

### **Identification**

Each product is identified with a lot number.

### **Quality**

Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501: *Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.*

### **Regulatory**

**FDA:** class I medical device exempt from 501(k).

**CE marking:** Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

### **Risks/Safety**

Slight risk of ocular or respiratory irritation in case of extended contact with the super absorbent pad. Safety data sheet is available upon request.

### **Handling/Storage**

Store in a clean and dry place, away from sun.

### **Packaging**

Unit	Box of 40 garbage bags	20 x 11.4 x 11.4 cm / 1 kg 8 x 4.5 x 4.5 in / 2.4 lbs
Case	10 boxes of 40 bags (400)	26.7 x 21.6 x 59.7 cm / 12 kg 10.5 x 8.5 x 23.5 in / 26.4 lbs
Pallet	40 cases per pallet or 400 boxes of 40 garbage bags	122 x 107 x 120 cm / 501 kg 48 x 42 x 47.25 in / 1106 lbs

- To order our super or extra absorbent pad, contact Hygie customer service at 1 866 588 2221