



X10



X20

ProHy21[®] URINAL Kit

Tender description

Preassembled kit containing Hygienic Covers and Hy21 urinal supports

Hygienic Cover :

- Absorption : 600 ml
- No expiry date

Eco-friendly product

Made in Canada 

Supports :

- Bedpan washer compatible
- Recyclable

Product code

Case code: KT-PRO-HY21-UL0

Présentation

This kit is designed for efficient management of bodily fluids.

It includes 200 hygienic covers[®] – urinal bags (10 boxes of 20) containing a super absorbent pad and 20 transparent Hy21 urinal supports.

Dimensions

Cover: Length = 14.96" (38 cm). Width = 6.3" (16 cm). Thickness = 38µm.

Support: Length: 5.1" – (13cm) x Width: 3.7" – (9.5cm) x Height: 9.6" – (24.4cm).

Composition

Polypropylene urinal support. Polyethylene bag equipped with ties. Super absorbent pad 100 % recycled composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. Packaging made of recycled materials. No latex and no phthalates.

Shelf life

No expiry date.

Efficiency

Hygienic Cover: absorption of 600 ml of liquids in less than 30 seconds.

Manufacturing process

Support: injection moulding.

Hygienic Cover: heat welding of a plastic film with insertion of ties.

Use

Hospital use.

Instructions for use

Refer to the tutorial for more detail.

Precautions for use

Support: single-patient use. It is recommended to discard the support after the patient's departure. This recommendation is essential in the case of an infectious patient.

Hygienic Cover: single-use. Discard after use. Keep away from the reach of children.

Cleaning / Sterilization

The urinal support must be regularly cleaned and when soiled. It is recommended to use a disinfectant detergent or an enzyme product associated with a mechanical rubbing action. Follow the instructions on disinfectants to maximize their efficiency (dilution, contact time ...). It can also be cleaned in a bedpan washer without exceeding the temperature of 203°F or 95°C.

Disposal

Dispose of product in regular waste or according to the protocol of the institution. Don't dispose the bag or the pad in toilet. The support is recyclable (Plastic #5 - PP).

Benefits

Increases hygiene, saves time, reduces odours, simple and risk-free disposal. Support is patented.

Identification

Each product is identified with a lot number.

Quality

Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501: *Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.*

Regulatory

FDA: class I medical device exempt from 501(k).

CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

Risks / Safety

Slight risk of ocular or respiratory irritation in case of extended contact with the super absorbent pad. Safety data sheet is available upon request.

Handling/Storage

Store in a clean and dry place, away from sun.

Packaging

| | | |
|--------|---|---|
| Case | 20 urinal supports + 10 boxes of 20 urinal bags | 40.5 x 60 x 32 cm / 7.2 kg 16 x 12.5 x 23.75 in / 16 lbs |
| Pallet | 18 cases per pallet or 18 urinal Pro kits Hy21 [®] | 122 x 102 x 133 cm / 153 kg 48 x 40 x 52¼ in / 338 lbs |