



Hy21[®] Urinal support

Tender description

Hy21 Urinal support designed for Hygienic Covers
Bedpan washer compatible
Recyclable
Made in Canada 🇨🇦

Product code

Case code: HY-SUPP-URI3-000

Presentation

Polypropylene Hy21[®]urinal support.
 It is used in association with an Hygienic Cover[®] – urinal bag containing a super absorbent pad. It allows urine collection for men or women.

Colour

Transparent.

Support dimensions

Length: 5.1" - (13cm) x Width: 3.7" – (9.5cm) x Height: 9.6" – (24.4cm).

Composition

Polypropylene. No latex and no phthalates.

Shelf life

No expiry date.

Manufacturing process

Injection moulding.

Use

For any person, men or women, who requires a urinal, whether in a health care institution, at home or during a trip.

Instructions for use

Refer to the tutorial for more detail.

Precautions for use

Single-patient use. It is recommended to discard the urinal support after the patient's departure.

Cleaning / Sterilization

The urinal support must be regularly cleaned and when soiled. It is recommended to use a disinfectant detergent or an enzyme product associated with a mechanical rubbing action. Follow the instructions on disinfectants to maximize their efficiency (dilution, contact time ...). It can also be cleaned in a bedpan washer without exceeding the temperature of 203°F or 95°C.

Disposal

Dispose of product in regular waste or according to the protocol of the institution. The support is recyclable (Plastic #5 - PP).

Benefits

Increases hygiene, saves time, reduces odours, simple and risk-free disposal. Patented system.

Identification

Each case of urinal supports is identified with a lot number.

Regulatory

FDA: class I medical device exempt from 501(k).

CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

Risks / Safety

This product was evaluated and is not considered hazardous based on the hazard criteria established by the OSHA (Occupational Safety and Health Administration).
 Safety data sheet is available upon request.

Handling/Storage

Store in a clean and dry place, away from sun.

Packaging

Unit	Not packaged individually	24.4 x 13 x 9.5 cm / 93.3g 9.6 x 5.13 x 3.73 in / 0.2 lbs
Case	20 urinal supports per case	40.5 x 60 x 32 cm / 2.7 kg 16 x 12.5 x 23.75 in / 6 lbs
Pallet	18 cases per pallet or 360 urinal supports	122 x 102 x 133 cm / 158 kg 48 x 40 x 52¼ in / 72 lbs