

Hygienic Cover® for toilet bowl



Tender description

Hygienic Cover for toilet bowl with a super-absorbent pad

Absorption capacity : 600 ml

Cover dimensions : 28" x 18" – 71 x 45.7 cm

No expiry date

Eco-friendly product

Made in Canada 

Product code

Case code: SA-HYGI-PCWC-000

Presentation

Hygienic cover® – toilet bag containing a super-absorbent pad which gelifies bodily fluids and reduces odours. It allows regular toilets to be used as dry toilets with no flushing required. Protects all types of toilet bowls and buckets.

Cover dimensions

Length = 28"- (71 cm). Width = 18"- (45.7 cm). Thickness = 38µm.

Composition

Polyethylene bag equipped with ties. Super absorbent pad 100% recycled composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. No latex and no phthalates.

Shelf life

No expiry date.

Efficiency

Absorption of 600 ml of liquids in less than 30 seconds.

Manufacturing process

Heat welding with insertion of sliding ties.

Use

It is recommended for disposal of urine or stool contaminated with biocides, radioactive or chemotherapy substances. It is also ideal in situation where specimens need to be collected for analysis or simply when water is not available.

Instructions for use

1. Open the pack and take out the roll of bags.
2. Unroll the first bag and tear it along the cutting line.
3. Open the bag and verify the presence of the super absorbent pad at the bottom of the bag.
4. Lift the toilet seat
5. Install the bag on the toilet bowl and lower the toilet seat
6. Let the patient relieve himself/herself into the bag.
7. Wait for the gelification of the liquids (± 600 ml within seconds).
8. Close the bag, tie it and discard.

Refer to the usage instructions for more detail.

Precautions for use

Single-use product. Do not reuse. Keep away from children.

Disposal

Dispose of product in regular waste or according to the protocol of the institution. Don't dispose in toilet.

Benefits

Protection for all types of toilet bowls and buckets.

Increases hygiene, saves time, reduces odours, simple and risk-free disposal.

Limits the spread of pathogens. Single-use product.

Identification

Each product is identified with a lot number.

Quality

Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501 : *Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.*

Regulatory

FDA: class I medical device exempt from 501(k).

CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

Risks/Safety

Slight risk of ocular or respiratory irritation in case of extended contact with the super absorbent pad. Safety data sheet is available upon request.

Handling/Storage

Store in a clean and dry place, away from sun.

Packaging

Unit	Roll of 18 toilet covers	17.8 x 12.7 cm / 0.68 kg 7 x 5 in / 1.5 lbs
Case	20 rolls of 18 toilet covers (360)	60 x 27 x 42 cm / 18.7 kg 23.5 x 10.75 x 23.5 inches / 41.2 lbs
Pallet	20 cases per pallet or 400 rolls of 18 toilet covers	126 x 102 x 121 cm / 396 kg 49.5 x 40 x 47.75 inches / 874 lbs