



## Hygienic Cover® for Hy21® bedpan support / commode chair

### Tender description

**Hygienic Cover for bedpan and commode chair with a super-absorbent pad**

**Absorption capacity : 600 ml**

**Cover dimensions : 24.5" x 18" – 62.2 x 45.7 cm**

**No expiry date**

**Eco-friendly product**

**Made in Canada** 

### **Product code**

Case code: SA-HYGI-CLBI-000

### **Presentation**

Hygienic cover® – bedpan bag containing a super-absorbent pad which gelifies bodily fluids and reduces odours. It is used in association with the Hy21® bedpan support or with a standard bedpan or commode chair.

### **Cover dimensions**

Length = 24.5"- (62.2cm). Width = 18"- (45.7cm). Thickness = 38µm.

### **Composition**

Polyethylene bag equipped with ties. Super absorbent pad 100% recycled composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. No latex and no phthalates. Packaging made of recycled materials.

### **Shelf life**

No expiry date.

### **Efficiency**

Absorption of 600 ml of liquids in less than 30 seconds.

### **Manufacturing process**

Heat welding with insertion of sliding ties.

### **Use**

For any person, men or women, who needs a bedpan and/or commode chair whether it's in an institution or at home. Particularly recommended for isolation case.

### **Instructions for use**

1. Open the packet and take out a roll of bags.
2. Unroll the first bag and tear it along the cutting line.
3. Open the bag and verify the presence of the super absorbent pad at the bottom of the bag.
4. Place the bag on the Hy21® bedpan support, the bedpan or the commode chair.
5. Let the patient relieve himself/herself into the bag.
6. Wait for the gelification of the liquids (± 600 ml within seconds).
7. Close the bag, tie it and discard.

**Refer to the usage instructions for more detail.**

### **Precautions for use**

Single-use product. Do not reuse. Keep away from children.

### **Disposal**

Dispose of product in regular waste or according to the protocol of the institution. Don't dispose in toilet.

### **Benefits**

Protection for Hy21® bedpan supports, standard bedpans or commode chairs.

Increases hygiene, saves time, reduces odours, simple and risk-free disposal.

Limits the spread of pathogens. Single-use product.

### **Identification**

Each product is identified with a lot number.

### **Quality**

Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501: *Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.*

### **Regulatory**

**FDA:** class I medical device exempt from 501(k).

**CE marking:** Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.

### **Risks/Safety**

Slight risk of ocular or respiratory irritation in case of extended contact with the super absorbent pad.

Safety data sheet is available upon request.

### **Handling/Storage**

Store in a clean and dry place, away from sun.

### **Packaging**

Unit	Box of 20 bedpan liners	20 x 11.4 x 11.4 cm / 0.94 kg 8 x 4.5 x 4.5 in / 2.07 lbs
Case	20 boxes of 20 liners (400)	60 x 27 x 42 cm / 20.6 kg 23.5 x 10.75 x 23.5 inches / 45.5 lbs
Pallet	20 cases per pallet or 400 boxes of 20 bedpan liners	126 x 102 x 121 cm / 435 kg 49.5 x 40 x 47.75 inches / 960 lbs