FastPack® Testo Calibrator

For use in conjunction with the FastPack® IP and Fastpack® Testo Immunoassay and
the FastPack® IP System Analyzer

CAUTION: United States Federal law restricts this device to sale and distribution by or on the order of a physician, or to a clinical laboratory; and use is restricted to, by or on the order of a physician.

INTENDED USE

The FastPack® Testo Calibrator are intended to calibrate the FastPack® IP System when used for the quantitative determination of total testosterone in human serum.

SUMMARY AND PRINCIPLE

Quantitative assay calibration is the process by which a set of samples with known analyte concentrations (i.e. assay calibrators) is tested to measure the response. The mathematical relationship between the measured responses and the known analyte concentrations establishes the calibration curve. The FastPack® IP Analyzer must be calibrated by the user to ensure that it is properly adjusted for the particular lot of FastPacks that is being used. The FastPack® Testo Calibrator is used for this purpose.

PRODUCT INFORMATION

- Provided ready to use.
- Mix contents by gently inverting before use. Avoid bubble formation.
- Calibrator: 2.5 mL/vial. Liquid. Tris Buffer solution with protein stabilizers.
- Calibrator Card: 1
- Preservative: 0.25% ProClin® 300

WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use only.
- Do not pipette by mouth.
- Do not eat, drink, or smoke in designated work areas.
- Do not mix calibrators from different lots.
- After opening, calibrator is stable until the expiration date on the label when stored and handled as directed. Do not use calibrator beyond expiration date. Calibrator stability is based on a closed vial stability study.
- Avoid microbial contamination of reagent when removing aliquots from the bottle.
- Refer to the FastPack® IP Procedure Manual for calibration procedures.
- The components containing ProClin® 300 are classified per applicable European Economic Community (EEC) Directives as: Irritant (Xi). The following are appropriate Risk (R) and Safety (S) phrases for ProClin® 300:

R36/38 Irritant to eyes and skin.
R43 May cause sensitization by skin contact.
S28 After contact with skin, wash immediately with plenty of soap and water..
S37 Wear suitable gloves.

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© 2000 Qualigen, Inc. All rights reserved. Qualigen and FastPack are trademarks or registered trademarks of Qualigen, Inc.
All other trademarks are the property of their respective owners.

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