

RAPID DIAGNOSTICS

osom[®]

H. pylori Test

FOR THE DETECTION OF H. PYLORI ANTIBODIES



www.osomtests.com



FEATURES:

- ✓ **ACCURATE;** Sensitivity 95.9% versus biopsy/histology
- ✓ **FAST;** Results in 10 minutes or less
- ✓ **SAMPLE;** Test only requires 10uL of serum or plasma; 25uL of whole blood
- ✓ **TRAINING;** Online training and certification; www.osomtraining.com
- ✓ **CUSTOM CARE;** Technical support staffed with certified medical technologist

RAPID DIAGNOSTICS

www.osomtests.com

OSOM[®]

H. pylori Test

Sekisui Diagnostics, LLC (AMERICAS)

4 Hartwell Place Lexington, MA 02421

Phone: 800 332 1042

Fax: 800 762 6311

Email: osomrapidtests@sekisui-dx.com

ISO 13485 Certified

Sekisui Diagnostics (UK) Limited (INTERNATIONAL)


Liphook Way, Allington, Maidstone, Kent ME16 0LQ, UK

Phone: +44 1622 607800 Fax: +44 1622 607801

Email: info@sekisui-dx.com

ISO 9001 Certified, ISO 13485 Certified


TEST PROCEDURE / RESULTS

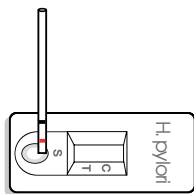

For serum or plasma,
fill capillary tube to
the red line.

For whole blood, fill
to the black line.



Apply sample by
lightly tapping the
tube on the pad.

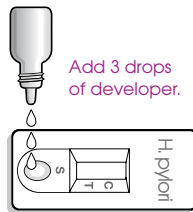


Add 3 drops
of developer.



Read results at
10 MINUTES.

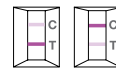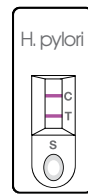

POSITIVE

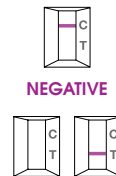

NEGATIVE

INVALID

DESCRIPTION	CONFIGURATION	CATALOG #	STORAGE	CPT CODE
OSOM [®] H. PYLORI TEST H. PYLORI CONTROL KIT	25 TESTS	175 (CE MARKED) 176	ROOM TEMP REFRIGERATED	86318QW (WHOLE BLOOD) 86677 (SERUM, PLASMA)

