

LiquiBand[®] Exceed[™] Topical Skin Adhesive has a unique, felt applicator tip to facilitate even, consistent application in three simple steps.

STEP ONE:

- While holding upright, squeeze the applicator wings until a cracking sound is heard.



STEP TWO:

- Invert the tip and gently squeeze wings to prime the device. When the tip turns violet, LiquiBand[®] Exceed[™] is ready to use.
- Ensure that wound is thoroughly clean and dry prior to closure.



STEP THREE:

- While holding the wound edges together, apply an even coat of LiquiBand[®] Exceed[™] over the length of the wound.
- Continue to hold the wound edges together for 60 seconds.



*If using on the eye area, adjust the patient's position. Use petroleum jelly to create a barrier close to the wound, and use surgical gauze to protect the patient's eyes.