









SETTINGS RECOMMENDATIONS

Tips \ Pure NEWTRON Units	POWER	IRRIGATION
---------------------------	-------	------------


PROPHYLAXIS

1 / 2 / 3 / 1S	14	
10P	14	
10X / 10Z	12	
EX1 / EX2 / EX3 / EXL / EXR	12	





PERIODONTICS

H1 / H2L / H2R / H3 / H4L / H4R	2	
P2L / P2R	3	
TK1-1S	2	
TK1-1L / TK2-1L / TK2-1R	2	

IMPLANT CARE






PH1 / PH2L / PH2R	2	
IP1	3	
IP2L / IP2R / IP3L / IP3R	5	

ENDODONTICS









CAP1	10	
CAP2 / CAP3	10	
ET18D	10	
ET20 / ET25 / ET25S / ETBD	7	


Tips \ Pure NEWTRON Units	POWER	IRRIGATION
---------------------------	-------	------------

ENDODONTICS


ET20D / ET25L / ET40 / ET40D	7	
IRR20-21 / IRR20-25 / IRR25-21 / IRR25-25	9	
K10 / K15 / K25 / K30	6	
AS3D / AS6D	7	
AS9D	6	
ASLD / ASRD	7	
P14D / S12-70D	7	
P15LD / P15RD	7	
SO4	7	

PROSTHESIS & ESTHETICS

PM1 / PMS1	15	
PM2 / PMS2	12	
PM3 / PMS3	10	
PM4	14	
PMV1 / PMV2 / PMV3	15	
PMV4 / PMV5 / PMV6	10	
5AE / ETPR	20	
C20	11	

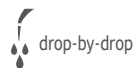

drop-by-drop

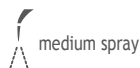

medium spray



high spray




no irrigation

ADJUSTMENT OF THE IRRIGATION FLOW IS ESSENTIAL

In order to obtain good drop-by-drop suitable for periodontal treatments, and a spray that does not create aerosol, the adjustment of the irrigation must be adapted to each tip:



1/ Set the irrigation flow rate to 0 and the power to 3 on the ultrasonic generator.

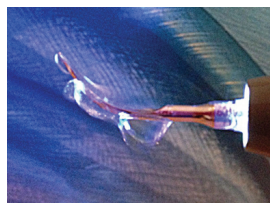

2/ Hold the handpiece with the tip pointing upwards.
3/ Adjust the spray by progressively increasing the flow rate so that the irrigation sprays the tip point with drop-by-drop.



4/ Set the machine to the required power.



5/ Start working with aspiration close to the tip.