


SORBACT® ABSORPTION DRESSING

Infection preventive absorbent dressing

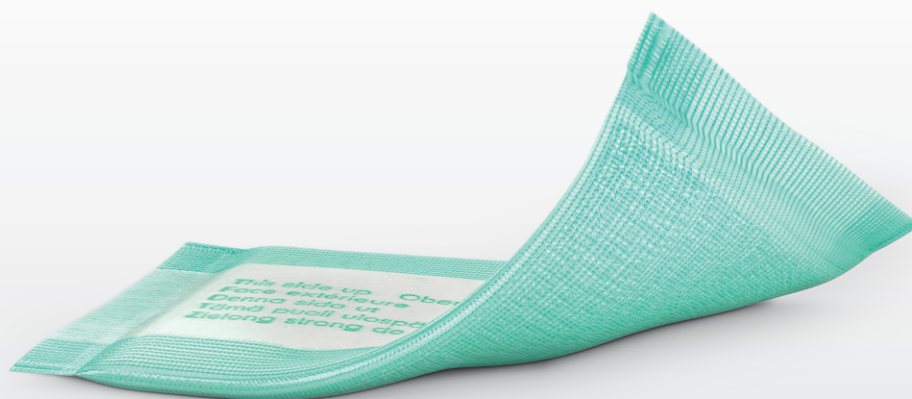

REDUCES BIOBURDEN IN MODERATELY TO HIGHLY EXUDING WOUNDS

Sorbact® Absorption Dressing is a bacteria and fungi binding wound dressing. It consists of a green Sorbact® wound contact layer combined with an absorbent core and a white backing. Sorbact® Absorption Dressing absorbs and retains exudate, thereby reducing the risk of maceration and enabling a moist wound environment. The dressing can be used in compression therapy.

Product	Ref. no.	Size	Pcs/pack
Sorbact® Absorption Dressing	98224	7x9 cm	20
Sorbact® Absorption Dressing	98222	10x10 cm	20
Sorbact® Absorption Dressing	98225	10x10 cm	10
Sorbact® Absorption Dressing	98223	10x20 cm	10




INTENDED USE

Sorbact® Absorption Dressing is intended for use in management of clean, colonized, contaminated or infected wounds with moderate to high exudate levels, such as surgical wounds, traumatic wounds, pressure ulcers, diabetic ulcers and foot and leg ulcers.

INSTRUCTIONS FOR USE

1. Prepare the wound according to local clinical practice.
2. Select an appropriate dressing size for the wound. The wound pad should overlap the wound margins by at least 2 cm.
3. Remove the dressing from the pouch using an aseptic technique.
4. Apply the dressing. Ensure that the green contact layer comes into direct contact with the complete wound surface.
5. Apply an appropriate dressing fixation.
6. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days.

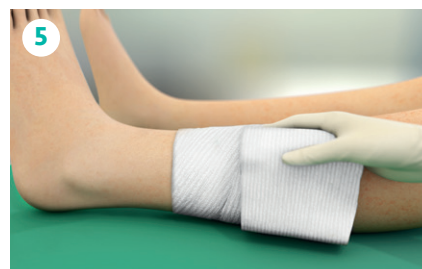

SORBACT® – INFECTION PREVENTIVE DRESSINGS

Sorbact® dressings employ a natural mechanism to bind microorganisms to their surface. The microorganisms are removed when the dressing is changed and the bioburden is thereby decreased.