


SORBACT® SUPERABSORBENT

Infection preventive high absorbency dressing

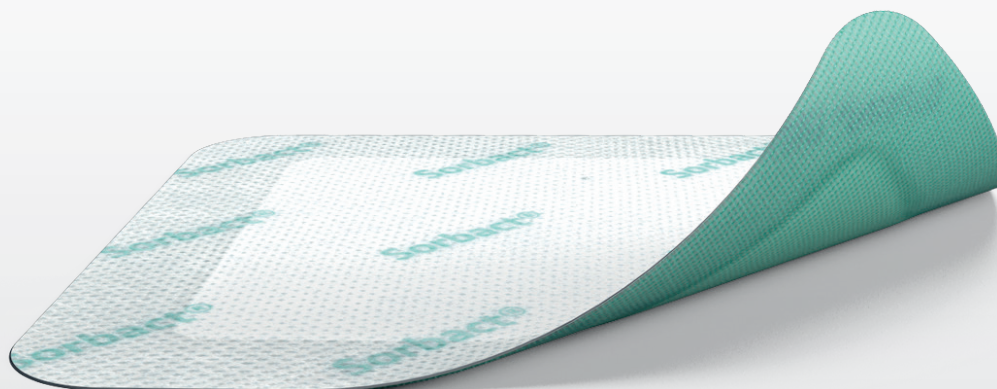

BIOBURDEN MANAGEMENT FOR HIGH TO EXCESSIVELY EXUDING WOUNDS


Sorbact® Superabsorbent is a sterile, bacteria and fungi binding, superabsorbent wound dressing. It consists of a green Sorbact® wound contact layer combined with a super-absorbent core and a white water repellent backing to prevent exudate strike through.

Product	Ref. no.	Size	Pcs/pack
Sorbact® Superabsorbent	98501	10x10 cm	10
Sorbact® Superabsorbent	98502	10x20 cm	10
Sorbact® Superabsorbent	98503	20x20 cm	10
Sorbact® Superabsorbent	98504	20x30 cm	10





INTENDED USE

Sorbact® Superabsorbent is intended for use in management of clean, colonized, contaminated or infected wounds with high to excessive exudate levels, such as surgical wounds, traumatic wounds, pressure ulcers, diabetic ulcers and foot and leg ulcers.

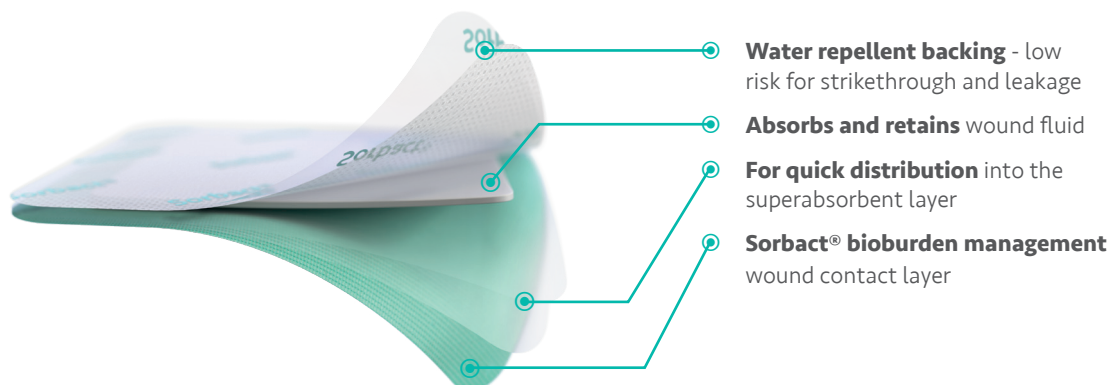

INSTRUCTION FOR USE

1. Prepare the wound according to local clinical practice.
 2. Select an appropriate dressing size for the wound. The wound pad should overlap the wound margins by at least 2 cm.
 3. Remove the dressing from the pouch using an aseptic technique.
 4. Apply the dressing. Ensure that the green side of the dressing comes into direct contact with the complete wound surface.
 5. Fixate with tape at dressing edges or cover with bandage. Ensure the dressing is allowed to expand under the fixation.
 6. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. If saturated the dressing shall be changed.
- Suited for use in compression therapy.

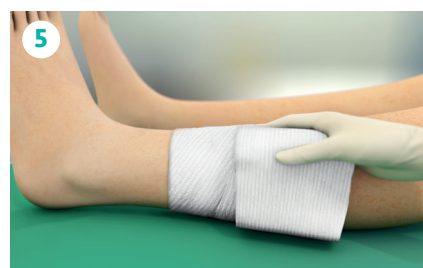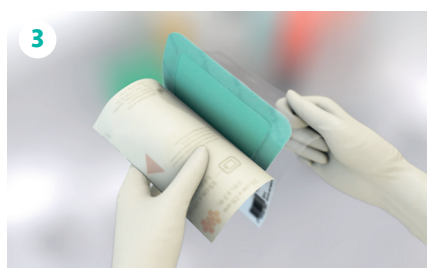

SORBACT® – INFECTION PREVENTIVE DRESSINGS

Sorbact® dressings employ a natural mechanism to bind microorganisms to their surface. The microorganisms are removed when the dressing is changed and the bioburden is thereby decreased.