

Excellent high-level disinfection from a
2.4% alkaline glutaraldehyde solution
with a 14-day reuse life.



CHECK OUT THE NEW BOTTLE SIZE!

ASP is thinking green! The new CIDEX[®] Activated Dialdehyde Solution bottle uses less plastic, almost 20% more solution per bottle—and that means less waste, which is better for the environment. Plus, the IFU is available online at www.e-ifu.com, which means less paper and more trees! Product code 2266 now replaces product code 2250.

ADVANCED STERILIZATION PRODUCTS

Division of Ethicon, Inc.

a *Johnson & Johnson* company

Trusted by hospitals the world over, CIDEX® Activated Dialdehyde Solution provides rapid, cost-effective, high-level disinfection for a wide range of endoscopes and other medical instruments. CIDEX® Activated Dialdehyde Solution offers excellent tuberculocidal activity combined with the noncorrosive advantages of glutaraldehyde. CIDEX® Activated Dialdehyde Solution is a 2.4% alkaline glutaraldehyde solution that provides excellent microbicidal and anticorrosion properties vs. acid glutaraldehydes.¹

EXCELLENT COMPATIBILITY

- Solution is compatible with a wide variety of medical instruments.

SIMPLE ACTIVATION

- Activate the CIDEX® Activated Dialdehyde Solution by adding the entire contents of the Activator vial, which is included with the CIDEX® Solution container.

RELIABLE

- Accurate, easy-to-use test strips detect the minimum effective concentration (MEC).
- Activated solution offers up to 14-day reuse life.

ACHIEVES HIGH-LEVEL DISINFECTION

- Broad-spectrum antimicrobial activity destroys all vegetative forms of bacteria as well as 6 log reduction of *Mycobacterium tuberculosis* in 45 minutes at 25° C.
- Effective in the presence of 2% organic soil.

SIMPLIFIES DISINFECTION

- After activating, just pour activated solution into compatible tray or automatic endoscope reprocessor and completely immerse instruments.
- Conveniently discarded down hospital and office drains in accordance with federal, state and local regulations.

ORDERING INFORMATION

REORDER#	DESCRIPTION	CASE CONTENTS
2266	Container of CIDEX® Activated Dialdehyde Solution	1.24 gallons (4.7L), 4 bottles
2920	CIDEX® Activated Dialdehyde Solution Test Strips	60 Strips/bottle, 2 Bottles

Please read and follow the CIDEX® Activated Dialdehyde Solution instructions for use prior to using this product for complete usage information, including contraindications and warnings.

